HARTMANN

Veroval[®]

duo control

EN – Upper arm blood pressure monitor

Instructions for use 3 – 39

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Warranty certificate/Garanti belgesi/

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Dear Customer,

We are pleased that you have chosen to purchase a blood pressure monitor from HARTMANN. Veroval® duo control is a quality product for fully automatic blood pressure measurement on the upper arm of adults and is suitable for both clinical and domestic use. Requiring no preliminary settings, this device easily and automatically inflates for simple, quick and reliable systolic and diastolic blood pressure and pulse rate measurement. The monitor also indicates possible irregular heartbeats.

The blood pressure monitor can be connected to a smartphone or tablet via *Bluetooth*®. On the smartphone/tablet you can evaluate the measured values using the Veroval® medi.connect app.

We wish you the best of health.

Please read these instructions carefully before using the monitor for the first time, since correct blood pressure measurement requires correct operation of the device.

These instructions will guide you through the individual steps for measuring your own blood pressure from first use of the Veroval® duo control upper arm blood pressure monitor. You will be given important and useful hints to ensure that you obtain reliable results for your personal blood pressure profile. Use this device in accordance with the information provided in the instructions for use. Please store these instructions carefully and ensure other users can access them. Check the packaging for any damage and that the contents are complete.

Package contents:

- Blood pressure monitor
- Upper arm cuff
- 4x 1.5 V AA batteries
- Storage bag
- Instructions for use with warranty certificate

DUO SENSOR Technology

The innovative DUO SENSOR Technology combines two professional measuring technologies: oscillometric technology and Korotkoff technology. Whilst most automatic blood pressure measurement devices only use oscillometric technology, DUO SENSOR Technology also utilises Korotkoff's very precise method of blood pressure measurement, which is also used by doctors to measure blood pressure. It is less error prone, providing correct measured values even for patients with heart rhythm disorders.

Healthcare professionals use a stethoscope to pick up the Korotkoff sounds and thus determine the blood pressure; Veroval® duo control works in the same way, using its integrated microphone.

The DUO SENSOR Technology thus produces accurate results while still being easy to use.

Comfort Air Technology

Comfort Air Technology enables the systolic blood pressure value to be approximated during inflation. This value is then used to determine the specific inflation pressure required for blood pressure measurement. This allows for more comfortable upper arm measurements.


Beyond statutory requirements, the device has been clinically validated in accordance with the ESH-IP2 protocol of the European Society of Hypertension (ESH), the protocol of the British Hypertension Society (BHS) and the protocol of the German society of hypertension (DHL).

Table of contents**Page**

1. Device and display description	6
2. Important information.....	8
3. Blood pressure information.....	15
4. Preparing the measurement	16
5. Measuring your blood pressure	18
6. Memory function.....	26
7. Transferring measured values via <i>Bluetooth</i> ® to the Veroval® medi.connect app	29
8. Explanation of error messages	31
9. Device care	33
10. Accessories.....	34
11. Warranty conditions.....	34
12. Contact information for customer enquiries.....	35
13. Technical data	36
14. Power supply unit	38
Warranty certificate	115

1. Device and display description

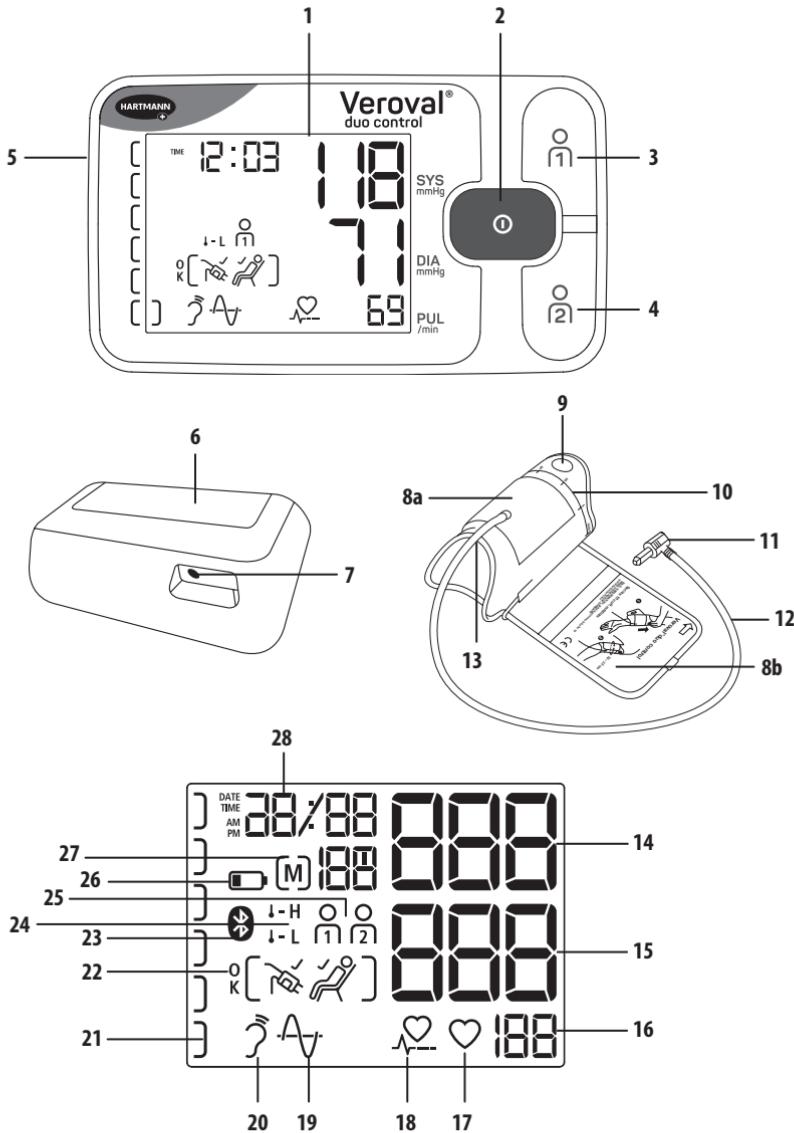**Blood pressure monitor**

- 1 Extra large LCD display with illumination
- 2 START/STOP button
- 3 Memory button for User 1
- 4 Memory button for User 2
- 5 Cuff connector socket
- 6 Battery compartment
- 7 Mains socket

Cuff

- 8 Secure fit cuff (a) with application instructions (b)
- 9 Grip tab to pull on the cuff
- 10 Size scale for correct adjustment of the cuff
- 11 Cuff plug
- 12 Extra long cuff tube
- 13 Lower arm opening for correct and secure fit

Display

- 14 Systolic blood pressure
- 15 Diastolic blood pressure
- 16 Pulse rate
- 17 Flashes when the monitor is measuring and the pulse rate is detected
- 18 Irregular heartbeat
- 19 Oscillometric measurement
- 20 Korotkoff measurement
- 21 Traffic light system for your values / progress bar
- 22 'Measurement OK' / cuff fit check / rest indicator symbols
- 23 *Bluetooth*® symbol
- 24 Temperature symbol
- 25 User memory
- 26 Battery symbol
- 27 Average value (A), morning (AM), evening (PM) / memory space number
- 28 Date and time display

2. Important information

Follow instructions for use

Medical Device

Manufacturer

Date of manufacture

Batch code

Catalogue number

Serial number

Caution (Please note)

Temperature limit

Humidity limitation

Air pressure limit

Protection against electric shock

Unique Device Identifier

Keep dry

Distributor

Importer

Direct current

IP20 Not protected against moisture, protected against solid objects ≥ 12.5 mm

Symbol for the marking of electrical and electronic equipment

Cardboard recycling code

Dispose of packaging in an environmentally responsible way

Dispose of packaging in an environmentally responsible way

With patented Secure fit cuff

With grip tab to pull on the cuff (Pull Up System)

Detects irregular heartbeat and measures correctly, even with heart rhythm disorders

Important instructions for use

Intended use:

The Veroval® duo control is a re-usable, non-invasive fully automatic blood pressure monitor for the upper arm that is intended for temporary monitoring of the systolic and diastolic blood pressure and the pulse rate of adults and which can be used by laypersons and healthcare professionals in clinical and domestic settings.

- Only use the monitor to take blood pressure measurements on the upper arm. Do not place the cuff on other areas of the body.
- Only use the supplied cuff or an original replacement cuff. The measured values may otherwise be incorrect.
- Only use the monitor on persons whose upper arm circumference is within the range indicated for the monitor.
- If measured values are questionable, repeat the measurement.

- Do not leave the device unattended near small children or persons who cannot operate it themselves. There is a risk of strangulation if the cuff tube becomes wrapped around the neck. Swallowing small parts that have detached from the device can also result in suffocation.
- Do not under any circumstances carry out blood pressure measurements on newborns, babies or toddlers.
- Please note that the pressure build-up in the cuff can lead to temporary disruption of medical devices being used at the same time on the same arm.
- The function of the arm on which the cuff is placed may become impaired during inflation.
- If you carry out the measurement on yourself or another person, please ensure that using the blood pressure monitor does not lead to ongoing reduction in the blood circulation.
- Taking measurements too often within a short time period as well as maintaining continuous cuff pressure can disrupt the blood circulation and cause injuries. Please take a break between measurements and do not bend or fold the air tube. If the monitor malfunctions, remove the cuff from the arm.
- If a possible allergic skin reaction occurs on the arm where the cuff is used, discontinue its use and consult a physician.
- Always consult your doctor about if and when the blood pressure monitor can be used by pre-eclampsia patients during pregnancy.
- The device is not intended for use in vehicles (e.g. ambulances) or helicopters.

Contraindications

- Do not place the cuff over a wound as this may cause further injuries.
- Do not place the cuff on persons who have had a mastectomy or lymph node clearance.
- If an intravascular treatment is present or there is an arteriovenous access (e.g. AV shunt) on the arm, measuring the blood pressure can lead to injuries. Never use the cuff on the arm to which these conditions apply.
- Do not use the blood pressure monitor in the absence of a full arm.
- If you are prone to hematoma, do not put on a cuff nor carry out a blood pressure measurement.

Important instructions for self-measurements

- Even minor changes in internal and external factors (e.g. deep breathing, stimulants, talking, excitement, ambient temperature) lead to fluctuations in blood pressure. This explains why deviating values are often measured at the doctor or pharmacist.
- Measurement results fundamentally depend on the measurement location and the position of the patient (sitting, standing, reclining). They are also affected by other factors such as exertion and the physiological condition of the patient. For comparable values, carry out the measurement in the same location and in the same position.
- Diseases of the cardiovascular system can result in erroneous readings or lower the accuracy of the measurement. Accuracy may also be affected if you have very low blood pressure, diabetes, circulatory disorders, arrhythmia, chills or tremors.

Consult your doctor before measuring your own blood pressure if ...

- you are pregnant. Blood pressure may vary during pregnancy. Regular blood pressure monitoring is particularly important if you have high blood pressure because elevated values may affect foetal development. Always check with your doctor to find out if and when you should measure your own blood pressure, especially if you suffer from pre-eclampsia.
 - you have diabetes, liver function disorders or narrowing of blood vessels (e.g. arteriosclerosis, peripheral arterial occlusive disease) or other pre-existing disorders or physical anomalies: in these cases, deviating measured values may occur.
 - you have certain blood disorders (e.g. haemophilia) or severe circulatory disorders or are taking blood-thinning medication.
 - you are on dialysis or taking anticoagulant medication (anticoagulants), platelet aggregation inhibitors or steroids.
 - you wear a cardiac pacemaker: this may lead to deviating measured values. The blood pressure monitor itself has no impact on cardiac pacemakers. Please note that the pulse rate displayed is not suitable for checking the frequency of cardiac pacemakers.
 - you tend to bruise easily and/or are sensitive to pressure pain.
 - you suffer from severe heart rhythm disorders or cardiac arrhythmias or atrial fibrillation (Afib).
- If this symbol  appears multiple times, it may indicate heart rhythm disorders or arrhythmia. Please consult your doctor in that case. Although the device is particularly well-suited for patients with heart rhythm disorders due to the DUO SENSOR measuring technology used, it is nevertheless possible that measurements may be incorrect in particularly severe cases (severe

heart rhythm disorders, arrhythmia and atrial fibrillation). Please discuss with your doctor whether taking your own blood pressure measurements is appropriate for you.

- Your self-measured values are for your information only – they are not a substitute for a medical examination! Discuss your measured values with your doctor and do not, under any circumstances, attempt to make your own medical decisions based on these measurements (e.g. on medications or their dosage)!
- Measuring your own blood pressure is not a substitute for medical treatment! Do not interpret your measured values yourself and do not use them to self-prescribed treatment.
Take measurements based on the instructions of your doctor and trust his or her diagnosis.
Take medication only as prescribed by your doctor and never alter the dose on your own accord.
Discuss with your doctor the appropriate time to measure your blood pressure.

An irregular heartbeat is present when the heart rhythm deviates by more than 25 % from the average heart rhythm. The contraction of the heart muscle is stimulated by electrical signals. A disorder of these electrical signals is called an arrhythmia. This may be due to a genetic predisposition, stress, age, lack of sleep, exhaustion or similar factors. A doctor can establish whether an irregular heartbeat is the result of an arrhythmia.

Heart rhythm disorders are disturbances of the normal rate or rhythm of the heartbeat. A distinction should be made between mild and severe heart rhythm disorders. This can only be determined by a special examination carried out by a doctor.

Power supply (batteries, power supply unit)

- Observe the polarity labels plus (+) and minus (-).
- Only use high-quality batteries (see specification in Chapter 13). If you use low-quality batteries, we can no longer guarantee the specified number of measurements.
- Never mix old and new batteries or batteries from different manufacturers.
- Remove flat batteries immediately.
- Replace the batteries if the battery symbol remains illuminated.
- Always replace all the batteries at the same time.
- If you do not intend to use the monitor for a longer period of time, you should remove the batteries to prevent possible leakage.
- If you use the monitor with the power supply unit, position the monitor so that you can disconnect the power supply at any time.

Information about batteries

- Choking hazard
Small children could swallow batteries and suffocate on them. Keep batteries out of the reach of children!
- Risk of explosion
Do not throw batteries into a fire.
- Batteries must not be charged or short-circuited.
- If a battery has leaked, wear protective gloves and clean the battery compartment with a dry cloth. If liquid from a battery cell comes into contact with skin or eyes, clean the affected area with water and seek medical attention if necessary.
- Protect batteries from excessive heat.
- Do not disassemble, open or crush batteries.

Safety information concerning the device

- This blood pressure monitor is not waterproof!
 - This blood pressure monitor is made of high-quality electronic precision components. The accuracy of the measured values and the lifetime of the monitor depend on careful handling.
 - Protect the device from mechanical influences, strong shocks, impacts or vibrations and do not drop it.
- Do not excessively bend or kink the cuff or the air tube.
- Never open the monitor. Do not modify, dismantle or repair the device yourself. Repairs may only be carried out by authorised specialists.
 - Do not inflate the cuff when it is not properly applied on the upper arm.
 - Use the device only with the approved upper arm cuff. The monitor may otherwise sustain external or internal damage.
 - The cuff tube may only be removed from the unit by pulling on the corresponding connector plug. Never pull on the tube itself!
 - Do not expose the device to extreme temperatures, humidity, dust, lint or direct sunlight as this may cause it to malfunction.
 - Keep the packaging, batteries and device out of the reach of children.
 - Protect the device and cuff against contact with pets and pests in order to avoid damage.

- Please comply with the storage and operating conditions defined in Chapter 13. Storing or using the device outside of the specified temperature and humidity range can affect measurement accuracy or the function of the device.
- If the device was not stored within the minimum/maximum permissible storage conditions, a waiting period of at least 2 hours must be observed before using it under the specified operating conditions (Chapter 13) or at an ambient temperature of approx. 20 °C.
- Do not use the device in explosive environments with inflammable gases or concentrated oxygen.

Instructions regarding electromagnetic compatibility

- Portable and mobile high-frequency and communication devices, such as telephones and mobile phones (including any accessories), can impair the functionality of this electronic medical device. Therefore keep a minimum distance of 30 cm.
- Do not use the device directly next to or between or stacked with other electronic products as this could lead to faulty operating.
- Do not use the device near strong electromagnetic fields and keep it away from radio equipment.
- Do not use the device together with a high-frequency surgical device or, for example, near magnetic resonance imaging equipment.
- The use of accessories other than those specified or provided by HARTMANN may result in increased electromagnetic emissions or reduced electromagnetic immunity of the device, resulting in incorrect operation.
- Further documentation in accordance with the electromagnetic compatibility standard can be requested from the manufacturer or the customer service (contact details see Chapter 12).

Instructions for the measurement function check

Every Veroval® device has been carefully tested by HARTMANN for measurement accuracy and has been developed with a view to a long service life. We recommend carrying out a metrological check every 2 years for **devices in professional use**, for example, in pharmacies, medical practices or hospitals. Please also observe the national regulations specified by the legislator. Metrological checks should only be carried out by the competent authorities or authorised maintenance providers at the user's expense.

Instructions for calibration mode

A functional check of the device can be carried out on a person or using a suitable simulator. A metrological check involves testing for leaks in the pressure system and potential deviations in the pressure reading. Remove at least one battery to switch to calibration mode. Now keep the START/STOP button ① pressed and re-insert the battery.

Keep it pressed for a few more seconds until the display shows a flashing 'L'. Then release the button and two zeros '0' appear on the display on top of one another. Instructions on the metrological check will be supplied by HARTMANN upon request to the competent authorities or authorised maintenance providers.

Disposal information

- To protect our environment, empty batteries must not be disposed of in household waste. Please comply with the relevant waste disposal regulations or use public collection points.
- This product is subject to the European Directive 2012/19/EU on Waste Electrical and Electronic Equipment and is marked accordingly. Never dispose of electronic equipment in your household waste. Please obtain information about your local regulations on the proper disposal of electrical and electronic products. This protects the environment and human health.

3. Blood pressure information

Two values must be measured to determine blood pressure:

- SYS – The systolic (upper) blood pressure: this is generated when the heart contracts and the blood is pumped into the blood vessels.
- DIA – The diastolic (lower) blood pressure: this is the pressure when the heart is relaxed and fills with blood again.
- The measured values for blood pressure are displayed in mmHg.

To better evaluate the results, there is a traffic-light-colour system on the left side of the Veroval® duo control that directly describes the results to enable easier categorisation of the measured value. The World Health Organisation (WHO) and the European Society of Cardiology (ESC) & European Society of Hypertension (ESH) have developed the following summary for classifying blood pressure values:

Results indicator	Evaluation	Systolic pressure		Diastolic pressure
Red	Grade 3 hypertension	above 179 mmHg	and/or	above 109 mmHg
Orange	Grade 2 hypertension	160–179 mmHg	and/or	100–109 mmHg
Yellow	Grade 1 hypertension	140–159 mmHg	and/or	90–99 mmHg
Green	High normal	130–139 mmHg	and/or	85–89 mmHg
Green	Normal	120–129 mmHg	and/or	80–84 mmHg
Green	Optimal	up to 119 mmHg	and	up to 79 mmHg

Classification of Office Blood Pressure and definition of Hypertension Grade
(Source: 2018 ESC/ESH Guidelines)

- Established hypertension (high blood pressure) is defined as having a systolic value of at least 140 mmHg and/or a diastolic value of at least 90 mmHg.
- Generally, blood pressure is described as being too low (hypotension) when the blood pressure value is less than 105 mmHg (systolic) and 60 mmHg (diastolic). However, the threshold between normal and low blood pressure (hypotension) is not as accurately specified as the threshold for high blood pressure (hypertension). Hypotension may be associated with symptoms such as dizziness, fatigue, tendency to faint, visual disturbances or high pulse rate. In order to ensure that hypotension or the associated symptoms are not signs of a serious illness, a doctor should be consulted in case of doubt.

Constantly elevated blood pressure significantly increases the risk of developing other diseases. Physical consequences such as a heart attack, stroke and organ damage are some of the most common causes of death worldwide.

Daily blood pressure monitoring is therefore an important measure that can help to protect you from these risks. If your blood pressure values are frequently elevated or borderline (see the table above), you should discuss this with your doctor. The Veroval® medi.connect app lets you share your readings easily with your doctor by email or as a print-out (see Chapter 7). Your doctor will initiate appropriate measures.

4. Preparing the measurement

Inserting/changing batteries

- Open the battery cover on the bottom of the device (see Fig. 1). Insert the batteries (see Chapter 13). Ensure correct polarity (+ and -) when inserting batteries. Close the battery lid. The device automatically switches to the date/time function, and when used for the first time, the display shows the number '3 /' for the day and '12' for the month. The date set is thus the 31st December. Set the date and time as described below.

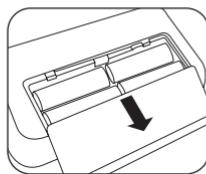
Fig. 1


- If the 'Change battery' symbol is permanently displayed, blood pressure can no longer be measured and you need to replace all the batteries.

Setting the date, time and Bluetooth®


! It is very important that you set the date and time correctly. This is the only way to save your measured values correctly, with the date and time, for subsequent retrieval. This is also important for correctly using all the memory and analysis functions.

To enter the setting mode, reinsert the batteries or press the START/STOP button ① for 5 seconds. Proceed as follows:

Date: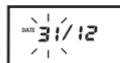

The number on the left (date indicator) on the display flashes. Press the ⌂ (+) or ⌂ (-) buttons to change the day displayed. For example, press ⌂ (-) twice to set the date to the 29th December. Save the current day by pressing the START/STOP button ①.

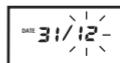
Now the number on the right, which displays the month, flashes. You can also set the month by pressing the buttons ⌂ (+) or ⌂ (-) and saving with the START/STOP button ①.


Now the year 2023 is displayed. You can also change the year displayed as described and confirm the change with the START/STOP button ①.

Time: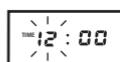

Then you can set the time. The number on the left flashes and shows 12:00. When the required hour is set, you can save this using the START/STOP button ①.

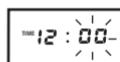Now the right number flashes. You can now change the display of the minutes and confirm the change with the START/STOP button ①.

The measured values will remain in the memory when the batteries are changed.
The date settings will also be saved, but the time settings must be reset.

Bluetooth®

The Bluetooth® symbol and is shown on the display.

Use the memory button or to select whether automatic Bluetooth® data transfer is activated (Bluetooth® symbol and is displayed, is flashing) or deactivated (Bluetooth® symbol and is displayed, is flashing) and confirm with the START/STOP button .

The device switches off automatically once all data have been set.

5. Measuring your blood pressure

5.1 The 10 golden rules of blood pressure measurement

Many factors play a role in blood pressure measurement. These ten general rules help you to correctly carry out the measurement.

1. Rest for about 5 minutes before measuring your blood pressure. Even working at your desk increases blood pressure by an average of about 6 mmHg systolic and 5 mmHg diastolic.

2. Do not consume any nicotine or caffeine up to one hour before taking the measurement.

3. Do not measure your blood pressure when you have a strong urge to urinate. A full bladder can increase blood pressure by about 10 mmHg.

4. Measure your blood pressure on a completely bare upper arm and in a comfortable upright seated position. The blood circulation must not be constricted by a rolled-up sleeve, for example.

5. If you are using a wrist monitor, hold the cuff at heart level during the measuring procedure. The cuff of an upper arm monitor is naturally positioned at the correct height on the arm.

6. Do not talk or move when measuring your blood pressure. Talking increases values by about 6–7 mmHg.

7. Wait at least one minute between taking repeat measurements so that the vessels are relieved from pressure in preparation for a new measurement.

8. The date and time are conveniently and easily recorded with the measured values as well as any medications you have taken, with the Veroval® medi.connect app.

9. Take measurements regularly. Even if your values have improved, you should continue to check them for monitoring purposes.

10. Always take measurements at the same time of day. As a person has about 100,000 different blood pressure values over the course of a day, individual measurements have no significance. Only regular measurements taken at the same time each day over a longer period of time enable the blood pressure values to be assessed meaningfully.

5.2 Applying the cuff

- Before applying the cuff, insert the cuff's plug into the connector socket on the left side of the monitor.
- Do not mechanically constrict, compress or bend the cuff tube.
- Blood pressure must be measured on a bare upper arm. If the cuff is completely open, thread the end of the cuff through the metal ring to form a loop. The hook and loop fastener must face outwards. Hold the cuff by the grip tab A (see Fig. 1) and pull it up over your upper arm.

Fig. 1

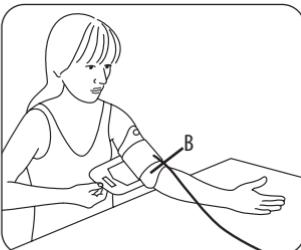
Fig. 2

- The opening B (see Fig. 2) on the cuff, opposite the grip tab, should lie on the inner elbow. The tube should lie in the middle of the inner elbow and point towards the hand.
- Now bend your arm slightly, grasp the free end of the cuff, wrap it firmly around your arm and close the hook-and-loop fastener.

⚠ The cuff should fit securely but not too tightly. You should be able to insert two fingers between the arm and the cuff. Make sure the tube is not bent or damaged.

⚠ Important: It is essential that the cuff is correctly fitted to obtain correct measurement results. The marking on the edge of the cuff helps you to choose the correct cuff size. The white arrow must point to an area within the sizing scale. If it is outside the scale, it is not possible to guarantee that the measurement will be accurate and a different cuff size is required (see Chapter 10).

This innovative Veroval® device with Comfort Air Technology provides comfortable measurement. The individual inflation pressure is determined independently for each measurement and depends on the corresponding systolic blood pressure value.

5.3 Carrying out the measurement

- You should take your blood pressure in a quiet place, in a relaxed and comfortable seated position.
- Measurement can be taken on the right or left arm. We recommend performing the measurement on the left upper arm. Over the long term, the arm giving higher measurements should be used for blood pressure monitoring. However, if there is a very clear difference between measurements on either arm, you need to check with your doctor which arm you should use for measurement.
- Always measure on the same arm and place your forearm in a relaxed position on a supporting surface.
- We recommend that you measure your blood pressure while sitting with your back supported by the back of the chair. Place both feet flat on the floor next to each other. Do not cross your legs. Place your forearm and hand, with your palm facing upwards, in a relaxed position on a supporting surface and make sure the cuff is on the same level as your heart.
- Do not measure your blood pressure after taking a bath or playing sports.
- Do not eat, drink or exercise for at least 30 minutes before the measurement.
- Please wait for at least one minute between two measurements.

5.4 Starting the measurement

The Veroval® duo control offers you two measurement modes. Depending on the situation and your preference, you can choose between regular single measurement (standard mode) or fully automatic multiple measurement (expert mode).

Standard mode (Single measurement)

- Start a measurement only after applying the cuff because the cuff may otherwise be damaged by the excess pressure that is created. Press the START/STOP button ①. The appearance of all display segments indicates that the monitor is carrying out an automated check and is ready for use.
- Check the display segments for completion (see picture in Chapter 1).
- After about 3 seconds, the cuff inflates automatically. If this inflation pressure is insufficient or if interference is experienced during measurement, the device will increase pressure to the appropriate higher pressure value in increments of 30 mmHg. During inflation, the results indicator on the left of the display also increases at the same time. (If no pulse is detected, the device inflates to a maximum of about 180 mmHg.)

If a higher inflation pressure is usually required, you can avoid re-pumping by holding down the blue START/STOP button ① again after the inflation process has started until the desired cuff pressure is reached. This should be about 30 mmHg above the systolic (upper) value.

- If the cuff has been applied firmly enough on the arm, the cuff symbol appears on the display. If the cuff symbol does not appear on the display, the cuff is not applied tightly enough and the error message 'Err - 2' may also appear on the display after a few seconds. Follow the instructions above as described in Chapter 5.2 for applying the cuff and the information in Chapter 8 and repeat the measurement.


Do not talk or move during the measurement.

- Three short beeps indicate that the necessary inflation pressure has been reached and that measurement is starting.
- As the air pressure in the cuff decreases, the heart symbol flashes and the dropping cuff pressure is displayed.

You can track the measurement progress using the progress bar. It increases during the inflation phase and decreases in the measuring phase. During the measuring phase, you will also see the symbols for the two DUO SENSOR Technology measuring methods. These indicate that the sensors are functioning correctly. In addition, the heart symbol shows your pulse, which is also measured.

- A long beep indicates the end of measurement. After the measurement is complete, the systolic and diastolic blood pressure values as well as the pulse rate (see Fig.) are shown on the display at the same time.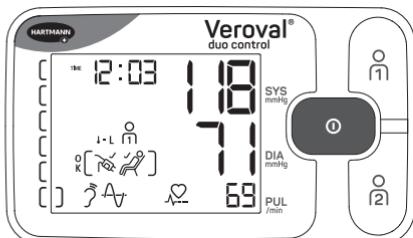

- The symbol  only appears on the display if the Veroval® duo control device does not detect any noticeable body movement during measurement, e.g., movement of the hand, arm or upper body. If the rest indicator symbol does not appear, the measurements may have been affected by body movement. Therefore, please follow the 10 golden rules (see Chapter 5.1) and repeat the measurement.
- The complete 'Measurement OK' symbol  only appears on the display if the cuff has been applied firmly enough and no noticeable body movement has been detected by the device during the measurement.
- If the symbols  -  or  -  appear on the display, the ambient temperature was too high or too low during the measurement, which can lead to deviating blood pressure values. In these cases, please follow the 10 golden rules (see Chapter 5.1) and repeat the measurement under optimal temperature conditions.
- In addition to the measured values, the time, the date, the associated user memory  or  are shown. The measured value is automatically assigned to the displayed user memory. While the result of the measurement is displayed, you can assign the values to the corresponding user memory by pressing the button  or . This assignment can be made for 10 seconds if Bluetooth® transfer is activated in the device settings. After this time the device starts the Bluetooth® data transfer automatically.
- You can classify your measured values using the results indicator on the left of the display (see the table in Chapter 3).
- To switch off the device, press the START/STOP button ; otherwise, the device will automatically switch off after 3 minutes.
- The measurement will not be saved if the power supply is interrupted before the device is switched off.

 If you wish to stop the measurement for any reason, simply press the START/STOP button . The cuff inflation or measurement process is interrupted and cuff pressure is automatically released.

- If this symbol  is shown on the display, the device has detected an irregular heartbeat during the measurement. However, the measurement may also have been disrupted by body movement or talking. It is best to repeat the measurement. If you see this symbol regularly when measuring your blood pressure, we recommend that your doctor checks your heart rhythm.

Expert mode (Multiple measurement)

- In expert mode, three blood pressure measurements are automatically taken in succession and the average value of the three measurements is displayed at the end.
- To start the fully automatic multiple measurement, press and hold the START/STOP button ① for 2 seconds until ‘3-’ appears on the display, then release the button.
- The information at the top left of the display shows you which of the measurements is currently running. The results of the individual measurements are not displayed.
- The time interval between the measurements is 60 seconds and is shown on the display as a countdown.

While the countdown is running, keep the cuff attached and remain seated at rest.

- After the last measurement, the final result is displayed. In addition, the ‘AB’ symbol and the date are displayed alternately at the top in the expert mode. This indicates that the value is calculated from the average of three measurements.
- The complete ‘Measurement OK’ symbol % [OK] appears on the display if during all three measurements the cuff has been applied firmly enough and if the user was sufficiently relaxed during the measurement.


The expert mode was developed in accordance with the recommendations of the international hypertension societies and the latest scientific findings. With regular use, it provides even better and more reliable blood pressure values for your personal health care and individual blood pressure monitoring.

Automatic Bluetooth® data transfer after measurement

If automatic Bluetooth® data transfer is activated, the Bluetooth® symbol starts flashing on the display 10 sec. after the measurement has been finished. The blood pressure monitor now attempts to connect to the Veroval® medi.connect app for approx. 2 minutes. The Veroval® medi.connect app must be active to allow data transfers and the monitor must have been successfully paired with the app (further details see Chapter 7).


As soon as a connection is established, all measurement data is automatically transferred to the app. During transmission the *Bluetooth®* symbol is displayed permanently and the results indicator on the left of the display runs from bottom to top.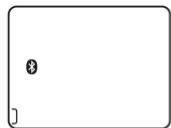

If you press the START/STOP button ① in the meantime, the transfer is cancelled and the device turns off.

Once all data has been transferred successfully, the result indicators are displayed permanently and the device switches off automatically after 5 seconds. If a connection to the app cannot be established after 2 minutes, the *Bluetooth®* symbol goes off and the blood pressure monitor switches off.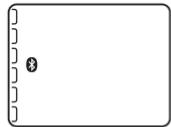


Operating in guest mode

If Veroval® duo control is used by a third person, it is recommended to use the guest mode. This mode ensures that a measured value is not saved to either of the two memory spaces (or).

This prevents distortion of the series of measurements and the average values of the two main users of the device.

In guest mode, both single measurement and multiple measurement (as described above in section „Expert mode“) can be performed.

To take single measurements in guest mode, start the measuring procedure by simultaneously pressing both memory buttons and . Do not press the START/STOP button ①. When the display with all segments is shown (see Chapter 1, fourth image), the initiation of the measurement was successful. During and after the end of the measurement, the two symbols and appear simultaneously on the display next to the measured values. The result can thus not be assigned to either person and the measured values will not be saved.

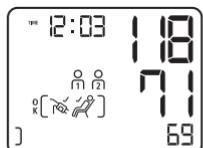To start the multiple measurement, simultaneously press and hold both memory buttons and until ‘- 3 -’ appears on the display.

To switch off the device, press the START/STOP button ①. Otherwise, the device will automatically switch off after 3 minutes.

6. Memory function

User memory

- Veroval® duo control stores up to 100 measurements in each user memory. The most recent measurement is always saved together with the date and time in memory slot no. 1 and all older saved values are moved up one memory slot. Once all memory slots are occupied, the oldest value is deleted.
- The device has two memory buttons $\textcircled{1}$ and $\textcircled{2}$, that are used to record the measurement results for two separate users. $\textcircled{1}$ represents the measured values for the first user, $\textcircled{2}$ represents the measured values for the second user. When a beep indicates the end of measurement, you can assign the measured value to the respective person by pressing $\textcircled{1}$ or $\textcircled{2}$. This assignment can be made for 10 seconds if *Bluetooth®* transfer is activated in the device settings. If you do not assign them, the measured value is automatically stored in the user memory that is currently displayed.
- The time of the measurement is also saved, along with each of the blood pressure values, to enable the morning and evening averages to be determined. The time saved in the device must therefore be set correctly to the actual time (see Chapter 4).

Veroval® duo control features the following measurement data storage functions (corresponding to the guidelines of the European Society of Hypertension (ESH)):

- Average value of all measured blood pressure values per user
- Average value of the morning values
- Average value of the evening values
- Individual measured value memory

Veroval® duo control follows the recommendations of the European Society of Hypertension (ESH) and differentiates between measurements taken in the morning and those taken in the evening. This distinction is medically relevant because blood pressure changes over the course of the day. With this information, your doctor is more likely to identify the right treatment for you if you should need medicinal treatment for high blood pressure.

- The memory recall is activated by pressing the $\textcircled{1}$ or $\textcircled{2}$ button with the device switched off. For values saved in the first user memory press the $\textcircled{1}$ button, for the second user memory press the $\textcircled{2}$ memory button.





Automatic Bluetooth® data transfer by memory function

- If automatic Bluetooth® data transfer is activated and the average value of all saved data is displayed, the Bluetooth® symbol  flashes on the display. The blood pressure monitor now attempts to connect to the Veroval® medi.connect app for approx. 2 minutes. The Veroval® medi.connect app must be active to allow data transfers and the monitor must have been successfully paired with the app. As soon as a connection is established, all measurement data is automatically transferred to the app. During transmission the Bluetooth® symbol  is displayed permanently and the results indicator on the left of the display runs from bottom to top. If you press the START/STOP button  in the meantime, the transfer is cancelled and the device turns off. Once all data has been transferred successfully, the result indicators are displayed permanently and the device automatically switches off after 5 seconds. If a connection to the app cannot be established after 2 minutes, the Bluetooth® symbol  goes off and the display keeps showing the memory data.

Average values



- After selecting the respective user memory, the corresponding symbol  or  and an 'R' appear on the display. The average value of all the stored data for the corresponding user memory is displayed.
- Pressing the  button again (or the  button if you are currently in user memory 2) will display the average morning measurement values 'RM' (00:00 to 11:59) for the past 7 days (including the current day).
- Pressing the  button again (or the  button if you are currently in user memory 2) will display the average evening measurement values 'PE' (12:00 to 23:59) for the past 7 days (including the current day).

If there is no morning or evening value for the past 7 days in the memory, dashes will be shown on the display instead of the morning or evening average values. If no values at all are stored in the memory, then dashes will also be displayed instead of the overall average value.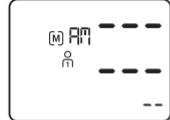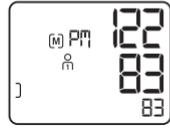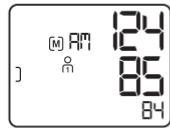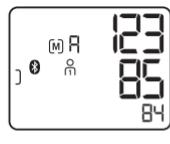

Individual measured values

- By pressing the button again (or the button if you are in user memory 2), all memory values can be retrieved one after the other, starting with the most recently measured value.
- In order to distinguish a single measurement from a multiple measurement, the '' symbol and the date/time are displayed alternately for a value that was measured in expert mode.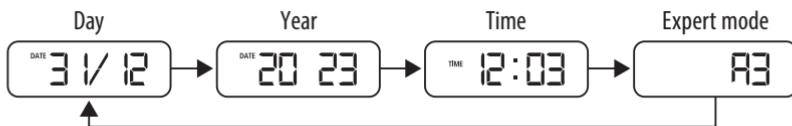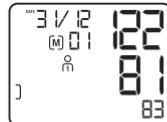


- If an irregular heartbeat or arrhythmia was detected during measurement, that information is also stored and displayed when the measured value is retrieved from the monitor's memory, along with the systolic and diastolic blood pressure measurements, pulse rate, time and date.
- The symbols , or (see Chapter 5.4) are also saved together with the individual measured value if they were displayed during the measurement.
- You can cancel the memory recall at any time by pressing the START/STOP button . Otherwise, the device will switch off automatically after 30 seconds.
- The saved values will still be available even if the power supply has been interrupted, e.g. due to changing the batteries.

Deleting saved measurements

All data saved for each user can be deleted separately for user memory and user memory . Press the button for the corresponding user memory (or). The average value '' is shown on the display.

Now hold the user memory button for 4 seconds until the display starts to flash, apart from the symbols and or . If you hold the memory button down for another 4 seconds, all data for the selected user memory will be deleted. To indicate this, dashes will be shown on the display instead of numerical values.

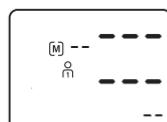The saved values can also be deleted when 'RM' or 'PM' is shown on the display, as described above. Again, all data for the selected user memory will be deleted.

Deleting individual values

If you would like to delete individual values, call up the relevant value and press the corresponding memory button (① or ②) for 4 seconds to make the display flash. After pressing the key for another 4 seconds, the individual value is deleted.

If you release the memory button early, no data will be deleted. When you delete an individual value, the next oldest measured value moves to the memory position of the deleted measured value. Deleting a measured value also recalculates the average values concerned.

7. Transferring measured values via *Bluetooth®* to the Veroval® medi.connect app

You can use the Veroval® duo control with your smartphone or tablet. You can transfer your data to your smartphone/tablet quickly and easily via *Bluetooth®* using the Veroval® medi.connect app.

Within the app, you can select different time periods and automatically receive perfectly prepared charts of your values. With a few clicks, all results can be printed or sent to the doctor via email.

Compatible devices

The Veroval® medi.connect app is designed and optimized for iOS and Android smartphones and tablets that support *Bluetooth®* 4.0 and run the following operating systems:

- Apple iOS 10 and higher
- Android 5.0 and higher

Android and iOS devices have subtle differences in their electronic components which can affect functionality, such as *Bluetooth®* connectivity. Due to the variation between device manufacturers, HARTMANN cannot guarantee optimal performance of the Veroval® medi.connect app across all available devices. A detailed list of compatible devices is available on our website at www.veroval.info.

Download and install the Veroval® medi.connect app

- To be able to use the Veroval® duo control with the app, you must download the app and install it on your smartphone or tablet.
- Download the Veroval® medi.connect app from the Apple App Store or Google Play Store. You can also download the app from our website www.veroval.info. To avoid lengthy download times and any associated costs, it is recommended to use a WiFi connection for the download.
- Install the app by following the step-by-step instructions in the Veroval® medi.connect app.

Connect the Veroval® duo control to the app

Step 1: Make sure that you have activated *Bluetooth®* on the blood pressure monitor (see Chapter 4, section '*Bluetooth®*').

Step 2: Make sure that you have activated *Bluetooth®* on your smartphone/tablet and that you have installed the Veroval® medi.connect app.

Step 3: Open your Veroval® medi.connect app on your smartphone/tablet. To allow data transfer make sure the app is active before you proceed with the following step.

Step 4: Perform a blood pressure measurement (see Chapter 5). The result is shown on the display and the *Bluetooth®* symbol flashes. Do not switch off the blood pressure monitor.


or **Step 4:** Data transfer at a later point:
Go to memory mode (see Chapter 6).
Select the desired user memory.

The blood pressure monitor now attempts to connect to the Veroval® medi.connect app for approx. 2 minutes.

Step 5: The *Bluetooth®* transfer starts automatically.

When connecting for the first time, a randomly-generated six-digit PIN code (i.e., 'pairing code') is displayed on the device, and at the same time an input field appears on the smartphone/tablet in which you must enter this six-digit PIN code. After successfully entering the code, the device will be connected (i.e., 'paired') to your smartphone/tablet.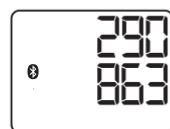

If your smartphone or tablet has a protective cover, remove it to ensure that there is no interference during the transfer.

The Veroval® medi.connect app is not intended for medical or commercial use and is no medical device in accordance with Regulation (EU) 2017/745.

The values displayed are purely for visualisation purposes and must not be used as a basis for therapeutic treatments. The software is not part of a diagnostic-medical system.

Visit www.veroval.info for a detailed Veroval® medi.connect tutorial.

8. Explanation of error messages

Error	Possible causes	Remedy
Monitor cannot be switched on	Batteries are missing, inserted incorrectly or flat.	Check the batteries. If necessary, replace with four new identical batteries.
	Power supply unit not correctly connected or faulty.	Make sure the power supply is connected to the socket on the back of the device.
Cuff will not inflate	Cuff connector plug is not positioned correctly in the socket of the monitor.	Check connection between cuff plug and connection socket.
	Wrong cuff type connected.	Check that only the approved Veroval® duo control cuff and the corresponding connector were used.
	The measuring signals could not be detected correctly or at all. Too few pulse beats or pulse too weak. Unusual measurement time or measurement results. Measurement result outside measuring range. General measurement error.	Check whether the cuff has been applied correctly. Do not talk or move during measurement.

Error	Possible causes	Remedy
	<p>Cuff does not inflate or does not inflate fast enough.</p> <p>The cuff was applied too firmly or too loosely.</p> <p>Unusual function of the pressure sensor.</p>	Apply the cuff so that two fingers fit between the cuff and the upper arm.
	<p>Air is released too fast or too slowly during the measuring procedure.</p> <p>The cuff could have become detached or loosened.</p> <p>Movement during the measuring procedure could also be the reason.</p> <p>Error during air release or unusual reduction in pressure.</p>	Check whether the cuff has been positioned correctly. Do not move during the measurement.
	Air tube has not been inserted into the device correctly or is kinked.	Check that the connector has been fitted correctly and ensure that there are no kinks in the air tube.
	<p>The inflation pressure is higher than 300 mmHg.</p> <p>The pressure is released automatically.</p>	Repeat the measurement after at least 1 minute of rest.
	<p>The data could not be sent via Bluetooth®.</p> <p>Bluetooth® was disconnected during data transfer.</p> <p>The device received a disconnect command from the app.</p> <p>Command wait timeout during data transfer.</p>	<p>Proceed as described in Chapter 7.</p> <p>Check if the device has been successfully paired with your smartphone/tablet in advance.</p> <p>Turn the device on and off and try again to establish the connection between device and smartphone/tablet.</p>

Error	Possible causes	Remedy
	If the battery symbol is flashing, the batteries are almost empty. Only a few more measurements can be taken.	Have new batteries of the same type ready (AA/LR06).
	If the battery symbol is permanently displayed, the batteries are empty and must be replaced.	Insert new batteries of the same type (AA/LR06). Then check the date/time and, if necessary, reset them (see Chapter 4).
Implausible measured values	Implausible measured values often occur due to inappropriate handling of the device or if there are mistakes in the measuring procedure.	Please note the 10 golden rules for blood pressure measurement (see Chapter 5.1) as well as all the important instructions in Chapter 2. Then repeat the measurement. If implausible values still occur: please contact your doctor.

- Switch the monitor off if an error message appears.
- Check for all possible causes and follow the 10 golden rules (Chapter 5.1) as well as the instructions for self-measurement from Chapter 2.
- Relax for 1 minute and then repeat the measurement.

9. Device care

- Only use a soft, damp cloth to clean the monitor. Please do not use thinners, alcohol, cleaning agents or solvents.
- The cuff can be cleaned carefully using a slightly damp cloth and mild soap solution. Do not completely immerse the cuff in water.
- It is recommended to clean and disinfect the cuff regularly or after each use, especially when used by several users, to prevent infection. The cuff should be disinfected, particularly on the inside, by wiping with a disinfectant. To disinfect the device and the cuff, 70 % isopropyl alcohol solution (active substance: 2-propanol) can be used. Other suitable specific disinfectants are listed on our website (Chapter 12).
- To protect the device from external influences, you can store the device and the cuff in the storage bag, together with these instructions.

10. Accessories

- To ensure measuring accuracy, only use original HARTMANN accessories which can be obtained from your pharmacist or specialist medical supplier.
- Accurate measured results cannot be guaranteed outside the upper-arm circumference ranges specified here.

Standard cuff, medium, for upper arm circumference of 22 – 32 cm
Art. no. 925 531

Standard cuff, large, for upper arm circumference of 32 – 42 cm
Art. no. 925 532

- Mains power operation: on the back of the device there is a socket for the adapter for mains operation (output 6V DC/600 mA). Only use the Veroval® power supply unit (model no. 925 391). Otherwise, the function and measuring accuracy of the monitor cannot be ensured.

11. Warranty conditions

- We offer a 5-year warranty for this premium-quality blood pressure monitor from the purchase date and in accordance with the conditions listed below.
- Warranty claims must be made during the warranty period. The purchase date must be documented by a correctly completed and stamped warranty certificate or proof of purchase.
- Within the warranty period, HARTMANN will replace or repair free of charge any faulty device components caused by material faults or manufacturing errors. This does not extend the warranty period.
- This device is solely intended for the purpose stated in these instructions for use.
- The warranty does not cover damage resulting from improper use or unauthorised interference. Accessory parts that are subject to wear and tear (batteries, cuffs, mains adapter cables, etc.) are excluded from the warranty. Claims for compensation are limited to the value of the product; compensation for consequential damages or injuries is expressly excluded.
- In warranty cases, please send the monitor with cuff and, if applicable, the power supply unit, together with the fully completed and stamped warranty certificate or purchase receipt directly or via your dealer to the customer services department of your country.

12. Contact information for customer enquiries

ZA - HARTMANN South Africa
Epsom Avenue, Northriding,
2169 Johannesburg
www.veroval.info
Tel. +27 860 4278 6266
phzahelpdesk@hartmann.info

If necessary, please contact us at the relevant address above for any questions regarding initial startup, use and maintenance of the device, or to report an unexpected operating malfunction or incident.

For a patient/user/third party in the European Union and in countries with identical regulatory regime (Regulation (EU) 2017/745 on Medical Devices); if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorized representative and to your national authority.

Date of revision of the text: 2023-09-13, Version 4

13. Technical data

Product description	Automatic upper arm blood pressure monitor
Model	Veroval® duo control
Type	DC3-18
Measuring method	Korotkoff and Oscillometric
Display range	0–300 mmHg
Measuring range	Systole (SYS): 50–250 mmHg Diastole (DIA): 40–180 mmHg Pulse: 30–199 beats/minute
Display unit	1 mmHg
Technical measurement accuracy	Cuff pressure: ± 3 mmHg Pulse: $\pm 5\%$ of the displayed pulse rate
Clinical measuring accuracy	Corresponds to the requirements of DIN EN 1060-4 and DIN EN ISO 81060-2; Korotkoff validation method: Phase I (SYS), Phase V (DIA)
Operating mode	Continuous operation
Nominal voltage	DC 6V
Power supply	4x 1.5V alkaline manganese Mignon (AA/LR06) batteries or the optional Veroval® power supply unit
Battery capacity	approx. 950 measurements
Protection against electric shock	Internally powered medical electrical (ME) equipment (only when batteries are used). Applied part: Type BF <input checked="" type="checkbox"/> = Class II ME device (when using the Veroval® power supply unit)
Protection against harmful ingress of water or solid materials	IP20 (not protected against moisture, protected against solid objects ≥ 12.5 mm)
Inflation pressure	Min. 140 mmHg
Comfort Air Technology	Individual inflation pressure depends on systolic blood pressure +30 mmHg

Automatic switch-off function	3 minutes after the end of the measurement or after setting the date/time; otherwise 30 seconds
Cuff	Veroval® duo control cuff for upper arm, for the following arm circumferences: Medium draw-clamp cuff 22 – 32 cm, Large draw-clamp cuff 32 – 42 cm
Pressure release valve	Electronically controlled linear valve
Memory capacity	2×100 measurements with average of all measurements and morning/evening average for the past 7 days
Operating conditions	Ambient temperature: +10 °C to +40 °C; Relative humidity: 15 – 85 %, non-condensing; Air pressure: 700 hPa – 1060 hPa
Storage/transport conditions	Ambient temperature: -20 °C to +50 °C; Relative humidity: 15 – 85 %, non-condensing
Serial number (SN)	Inside the battery compartment
Service life (operating life)	5 years
Bluetooth® data transfer	2400 MHz – 2483.5 MHz frequency band Transmission power: +2.2 dBm The blood pressure monitor uses Bluetooth® low energy technology compatible with Bluetooth® ≥ 4.2 smartphones/tablets
Reference to standards	EN 60601-1; EN 60601-1-2

14. Power supply unit

Model no.	LXCP12-006060BEH
Input	100 – 240 V~, 50 – 60 Hz, 0.5 A max.
Output	6 V DC, 600 mA, only in conjunction with the Veroval® duo control blood pressure monitor
Manufacturer	Globalcare Medical Technology Co., Ltd.
Protection	The device is double-insulated and has a primary fuse which disconnects the monitor from the mains in the event of a fault.
	Polarity of the DC voltage connection
	Protective insulation / Protection Class 2
Housing and protective covers	The power supply unit housing prevents touching contact being made with parts that are carrying or can carry a current (fingers, needles, test hooks). The user must not touch the patient and the outlet plug of the AC/DC power supply unit at the same time.

Legal requirements and guidelines

- The device complies with international standard IEC 80601-2-30 (Medical electrical equipment Part 2-30: Particular requirements for the basic safety and essential performance of automated non-invasive sphygmomanometers).
- Clinical testing of the measuring accuracy was carried out in accordance with EN 81060-2. The requirements of ANSI/AAMI test protocol SP10-1992 are also fulfilled.
- We hereby confirm that this product complies with the European RED Directive 2014/53/EU. The CE Declaration of Conformity for this product can be found under: www.veroval.info

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth® SIG, Inc. and any use of such marks by PAUL HARTMANN AG is under license. Other trademarks and trade names are those of their respective owners.

Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc., registered in the U.S. and other countries.

Google Play and the Google Play logo are trademarks of Google LLC.

Android is a trademark of Google LLC.

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