Veroval[®]

Veroval®

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
Veroval®

The screenshot shows the Veroval® medi.connect web interface. At the top, there is a navigation bar with the following tabs: Prehľad, Krvný tlak, EKG, Hmotnosť, Teplota, Domáci test, Aktivita, and Nastavenia. Below the navigation bar, the 'Krvný tlak' section is active. It features a title 'Krvný tlak' and a 'Náhľad' (View) button with icons for a line graph, a pie chart, a table, and a PDF document. Below the view options, there is a date range selector showing 'Týždeň' (Week) selected, with dates '23.09.2017' and '29.09.2017'. There are also radio buttons for 'ráno' (morning), 'večer' (evening), and a checked checkbox for 'celý deň' (all day). A 'Hľadať' (Search) button is highlighted with a mouse cursor. To the right of the search button is a checkbox for 'MAD'.

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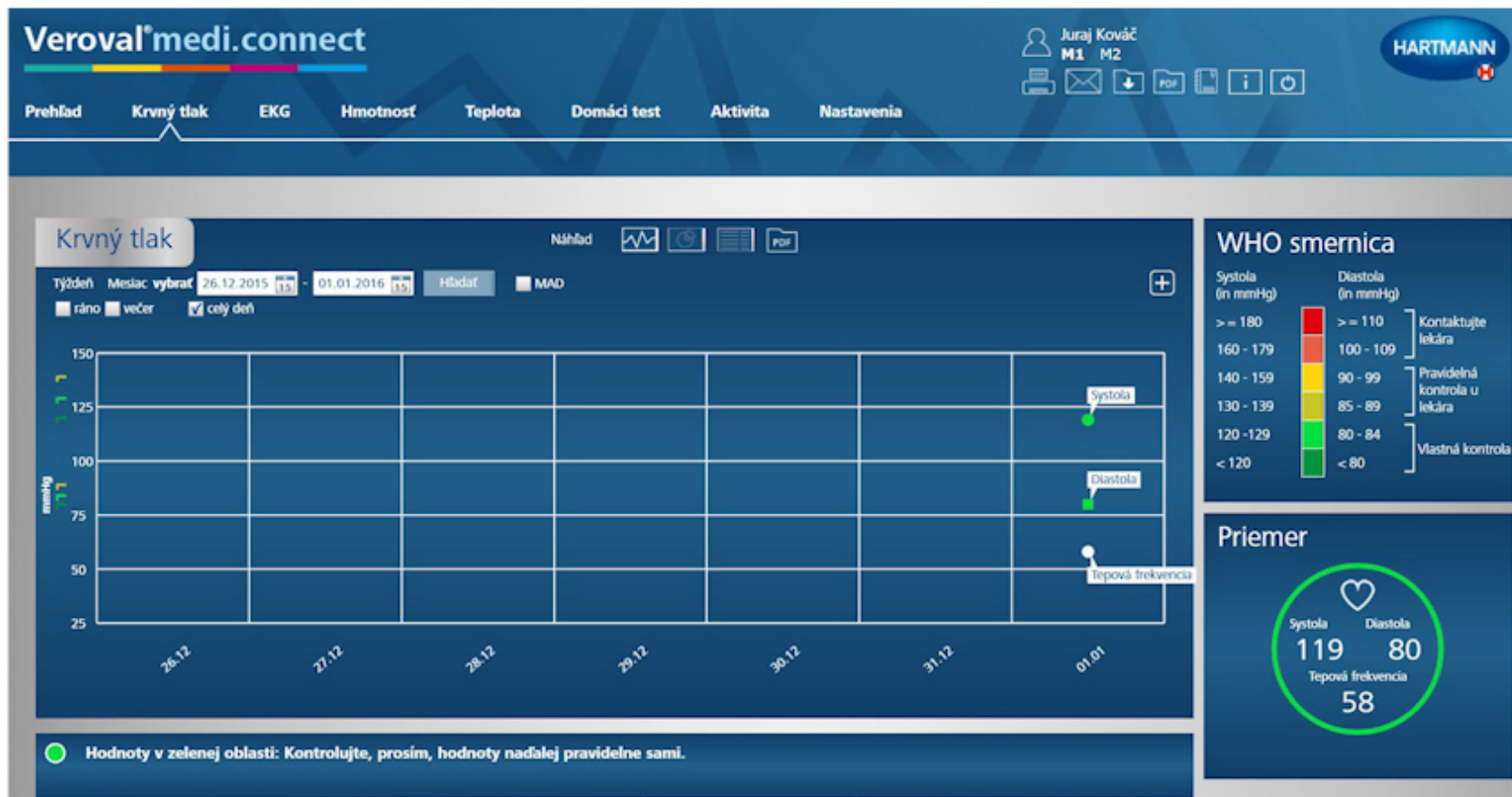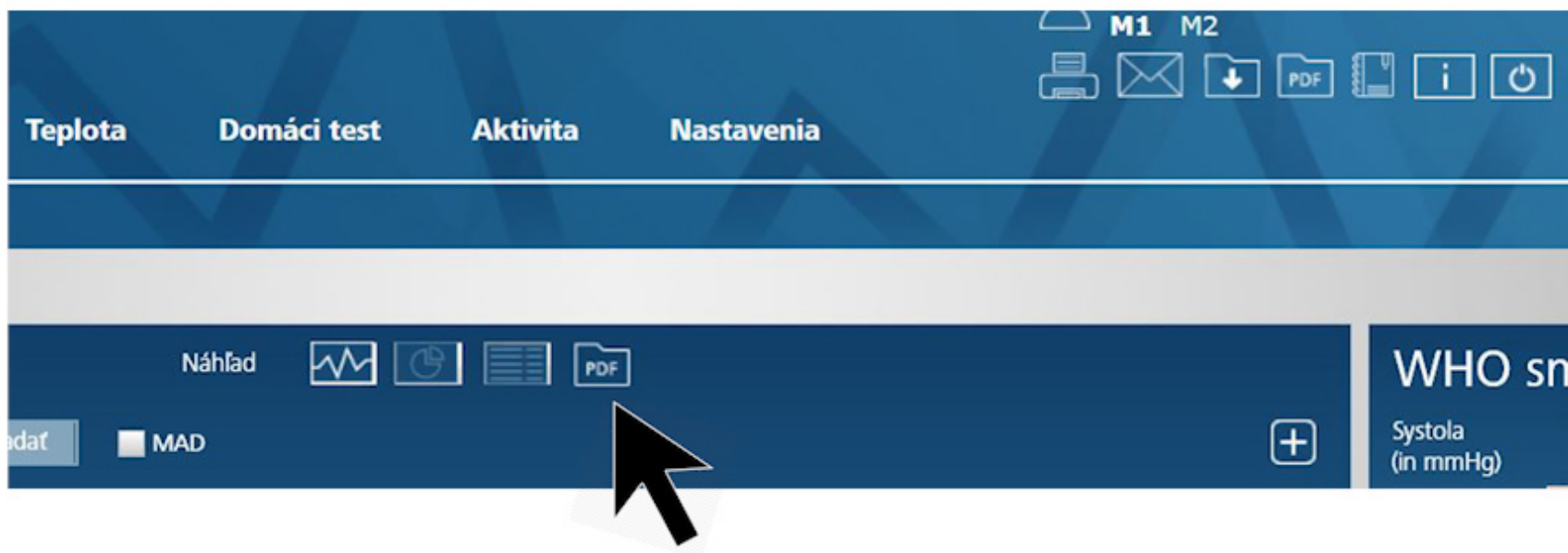
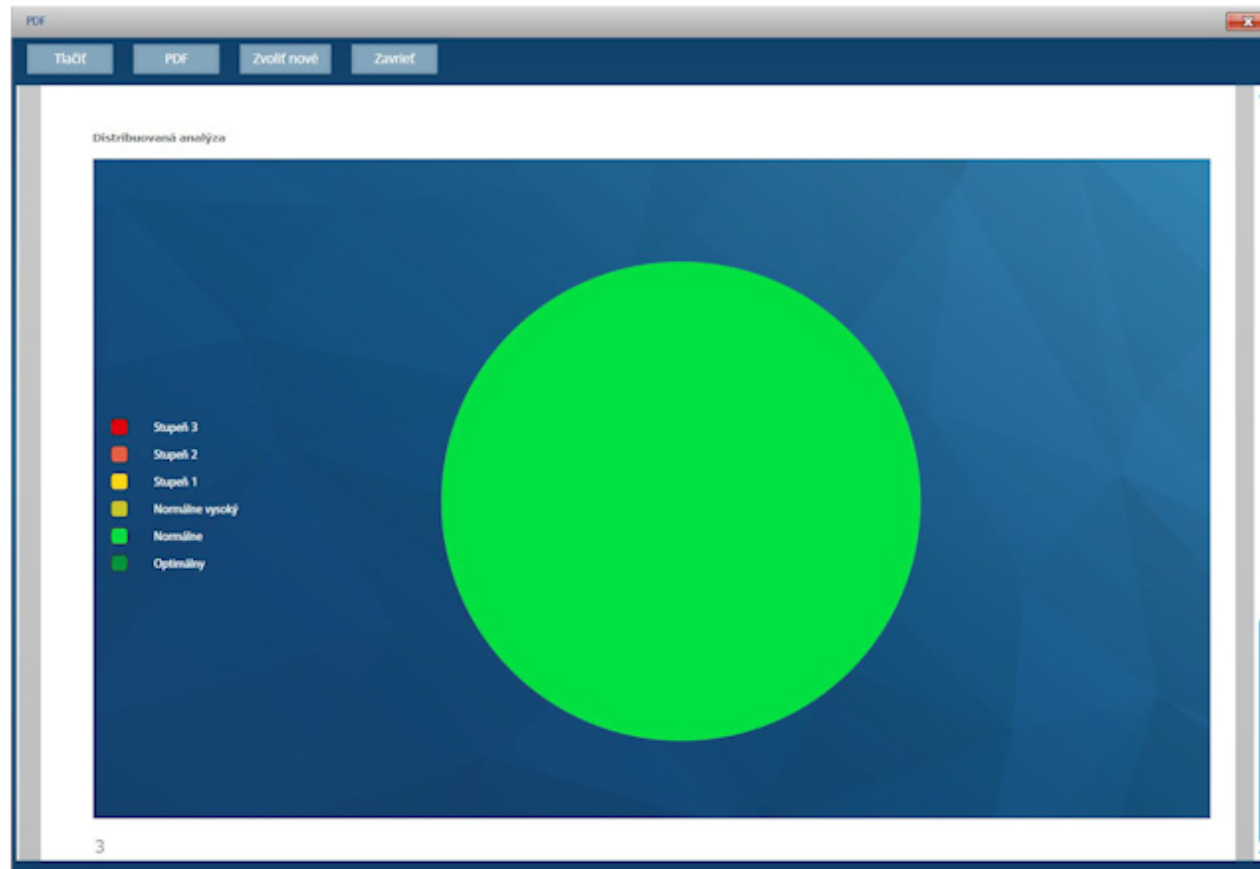
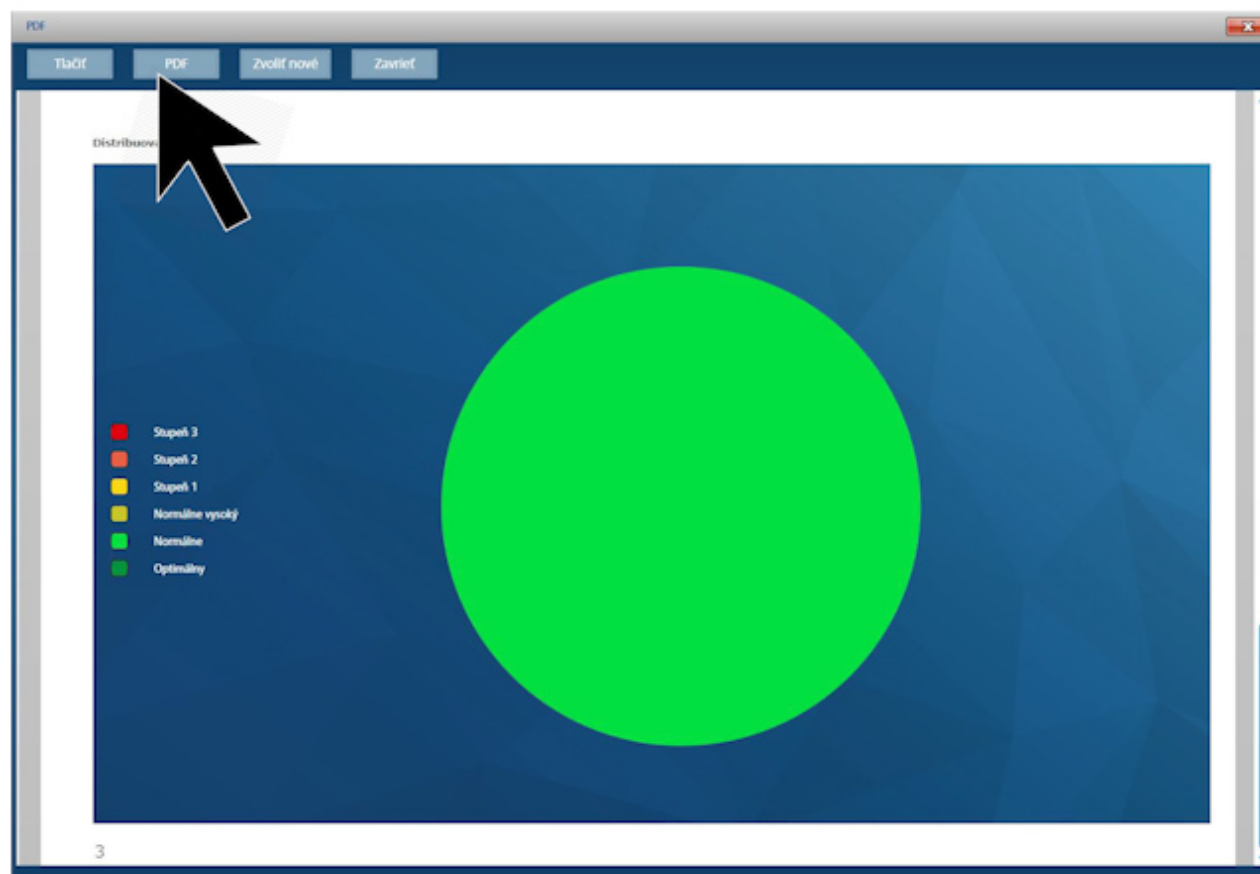
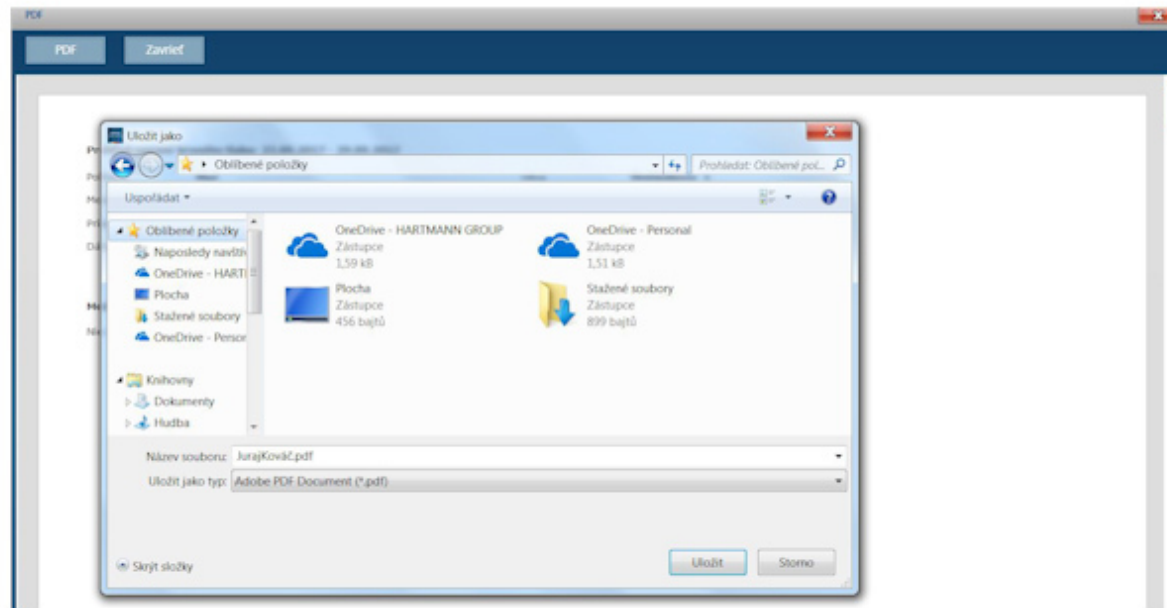
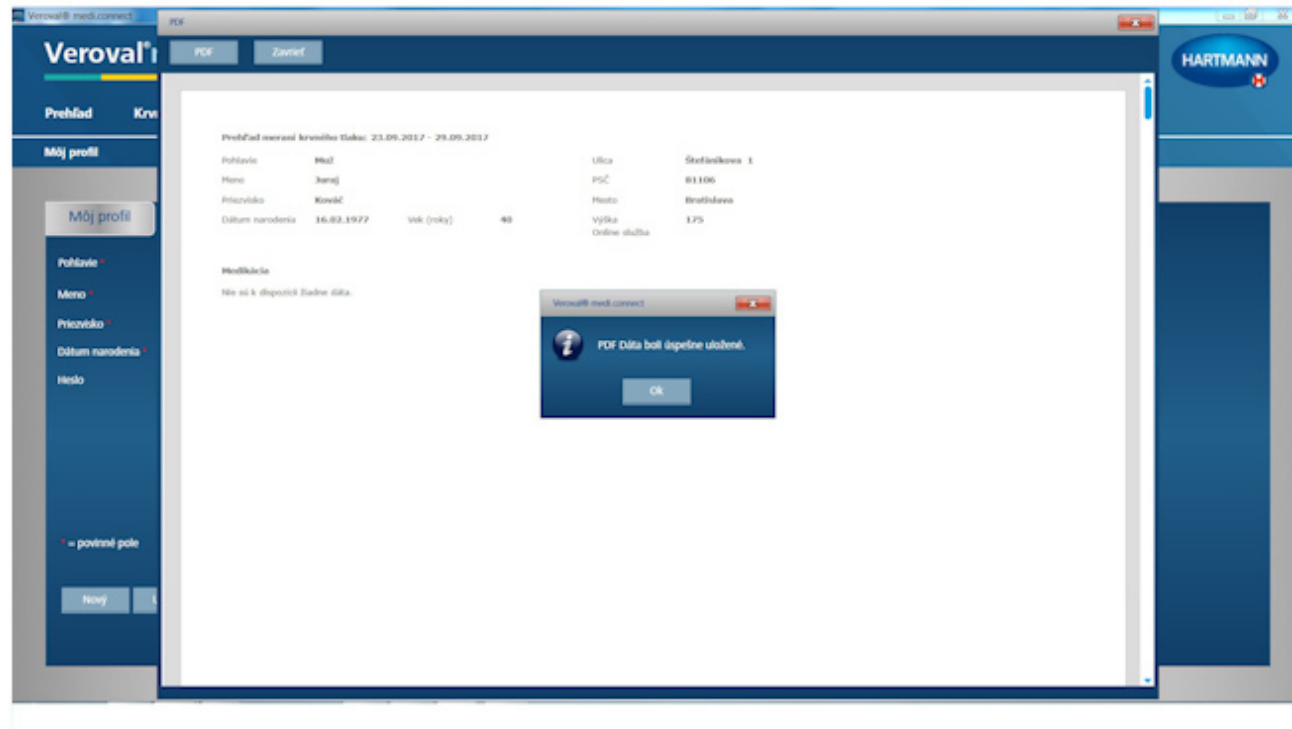
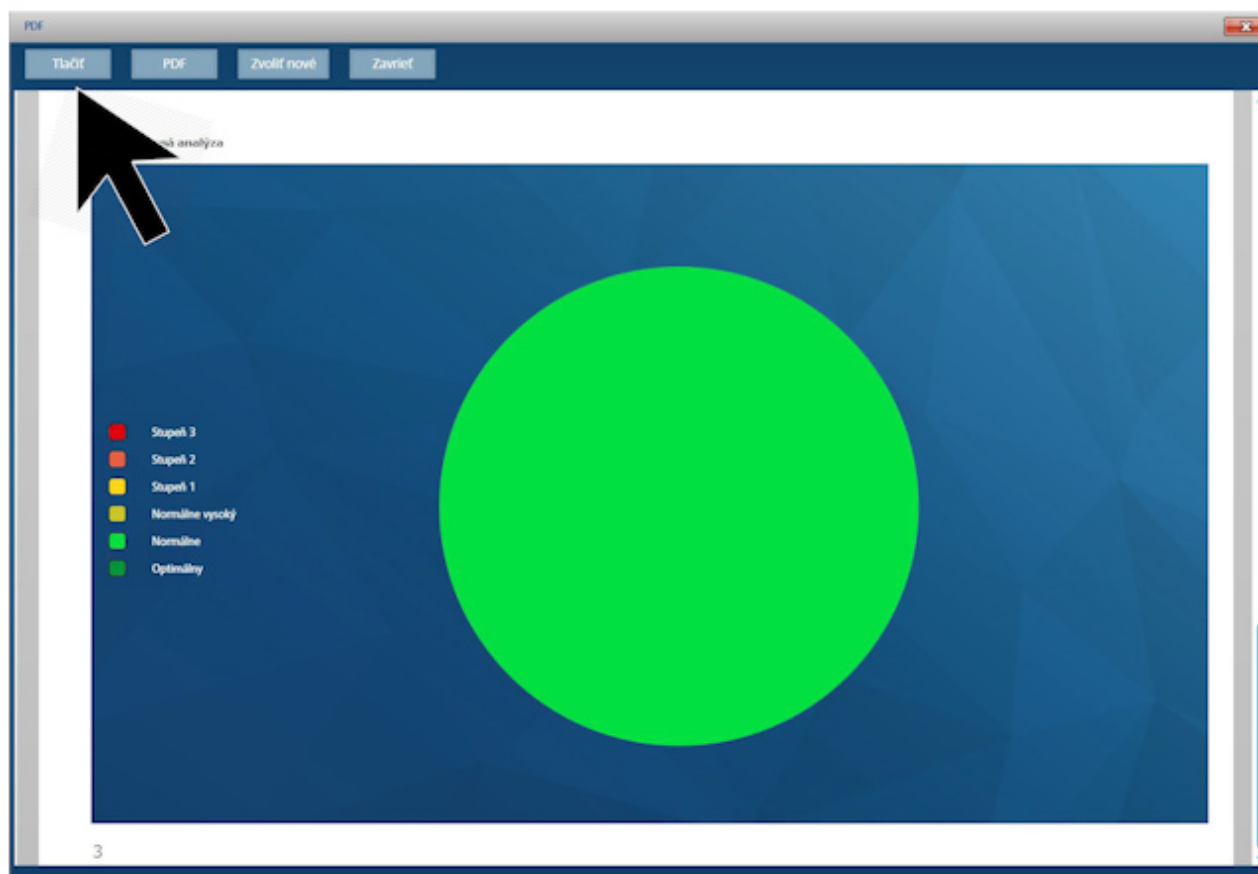
Veroval®

The screenshot displays the Veroval® medi.connect web application interface. At the top, the user is identified as Juraj Kováč, M1 M2, with the HARTMANN logo. The navigation menu includes Prehľad, Krvný tlak (selected), EKG, Hmotnosť, Teplota, Domáci test, Aktivita, and Nastavenia. The main content area is titled 'Krvný tlak' and shows a table of blood pressure readings for the week of 26.12.2015 to 01.01.2016. A single reading is visible: 119/80 mmHg with a heart rate of 58 bpm, marked as 'Vlastná kontrola' (green). To the right, the 'WHO smernica' (WHO guidelines) are shown with a color-coded scale: red (>= 180), orange (160-179), yellow (140-159), light green (130-139), green (120-129), and dark green (< 120). Corresponding diastolic ranges and actions are listed: >= 110 (Kontaktujte lekára), 100-109 (Pravidelná kontrola u lekára), 90-99 (Pravidelná kontrola u lekára), 85-89 (Pravidelná kontrola u lekára), 80-84 (Vlastná kontrola), and < 80 (Vlastná kontrola). Below the table, a note states: 'Hodnoty v zelenej oblasti: Kontrolujte, prosím, hodnoty naďalej pravidelne sami.' (Values in the green area: Please check, continue to check regularly yourself.)

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