Veroval®
baby

Developed
in
GERMANY

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The interpretation of the body temperature
depends on the place of measurement.

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Visual Fever Indicator

Alerts you if your child's
temperature is ≥ 37.6 °C
by the screen light-up in
RED color!

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VER067023

Facts about non-contact thermometers

Forehead thermometers measure the temperature of the body via the *arteria temporalis*. It emits nearly the same temperature as the core of our body.

Tips and advice

In case of fever, you may comfort your child by taking certain measures:

- Make sure your baby drinks enough. Offer him/her to drink every 30 min.
- Confine your baby to bed, rest is important for recovery.
- Do not bundle your baby in too many clothes and blankets to avoid building up heat.
- Check your baby's temperature at regular intervals.
- Consult with your physician.

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This list makes no claim to completeness.

Veroval[®] baby

As gentle as you

✓ **3 in 1 non-contact thermometer** on the forehead, object's surface and in the room

✓ **High measuring accuracy** in 3 sec

✓ **Soundless** to let your little one sleep

✓ **Positioning light** to guide your measurement during the scanning process

✓ **Visual fever alert:** display lights up in red when the forehead temperature is ≥ 37.6 °C

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

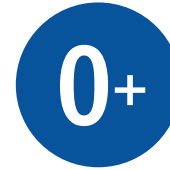
Suitable for ages 0+

The gentle and silent way for measurement

On forehead

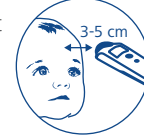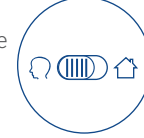1. Select the measuring mode "**forehead**" on the side of the device.
2. Press the on/off button **O/I** to switch on the device.
3. Place the thermometer at a distance of 3-5 cm at the center of the forehead.
4. Press "**SCAN**" and move the thermometer from the center of the forehead towards the temple.
5. Wait for 3 sec, until the display illuminates and you can read the result.

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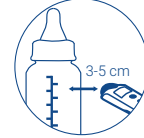
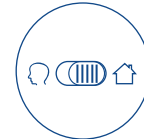
The new **Veroval[®] baby** also measures the temperature of milk bottles, bathwater and your baby's room...

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On objects and environment

1. Select the measuring mode "**home**" on the side of the device.
2. Press the on/off button **O/I** to switch on the device.
3. Place the thermometer at a distance of 3-5 cm from the object.
4. Press "**SCAN**" without moving.
5. Wait for 3 sec until the display illuminates and you can read the result.

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Veroval[®]
baby

The measuring distance is correct if the light forms a round point with circle around it.

The measuring distance is not correct if the light forms a misshaped point with blurry and dispersed circle around it.

Before measuring the temperature, the thermometer and the baby should be at the same room for at least 30 minutes.

Avoid measurement during eating, breastfeeding, drinking or activity times.

After the baby wakes up from sleep, wait for a few minutes, then measure the baby's temperature.

Before measuring the temperature, remove any hair, sweat, cosmetics or dirt from the forehead.

Wait for 30 seconds between two measurements.

Keep in mind that the temperature of the baby can be influenced by being outdoors for prolonged periods (e.g. in cold winter or in hot summer).

Do not measure temperature directly after showering, when the forehead is wet or after removing the baby's head covering.

Keep the sensor free from any residual dirt, scratches and not clouded by condensation especially when measuring boiling water or steaming bathwater.

Be careful, the temperature inside the feeding bottle or bathing may be warmer or colder than the temperature measured on its surface!

Do not store the device in an area exposed to direct heat or sunlight (e.g. car compartment, kitchen, and garage).

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