Veroval® medi.connect

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Easy data administration and storage

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Data transfer for Veroyal® Blood Pressure Monitors

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For tracking your values, start by setting-up the date and time on the device before using it.

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1. Select **Blood Pressure** in the software.

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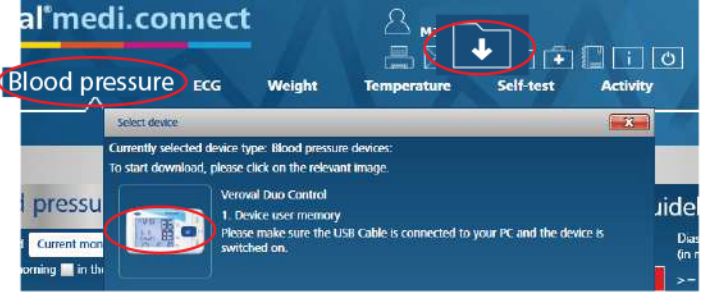
2. Plug in the USB Adapter into your PC and wait for the device to mention "PC" on the screen.

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3. Click on **Data Transfer** and select the Blood Pressure model (Duo Control / Upper Arm / Wrist). It will automatically transmit the recorded values to your computer.

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4. After transmission, select the values you want to store, then click **Save**.

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5. The stored values are now saved that you can download as PDF, print or email to your doctor.

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☒		Veroval upper arm blood pressure monitor	1. Device user memory
☐		Veroval Duo Control	2. Device user memory
☐		Veroval wrist blood pressure monitor	

Keep Track of your values using Veroval® MediConnect

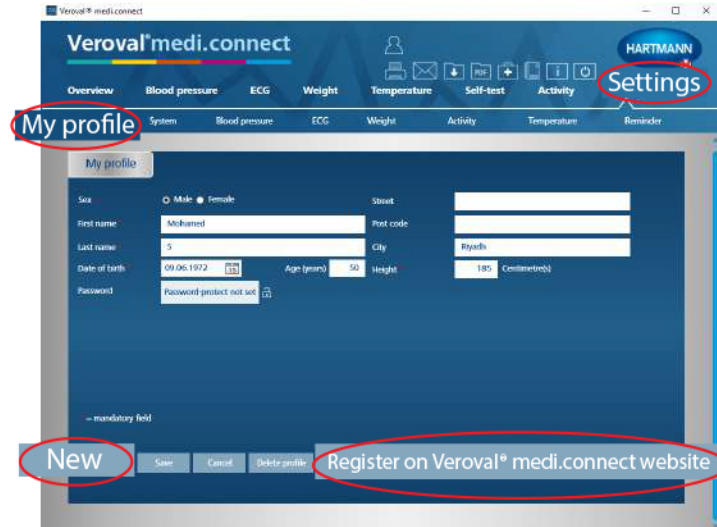
SOFTWARE* DOWNLOAD AND INSTALLATION

1. Visit www.veroval.info/en/mediconnect
Click on **Download Veroval® medi.connect desktop software** and install.
2. Once installation is complete, an icon will show on your computer's desktop.

ACCESS AND REGISTRATION

1. Double click on the **Veroval® medi.connect** software which will take you to the homepage.
2. Go to **Settings** then **My profile** and fill in your information: Gender, Age, Height. On the same page, fill in your desired password, then click Save.
3. Click on **Register on Veroval® medi.connect** with your email and password to be able to synchronize your values and access from any browser.

4. In Settings, select your device:
Blood Pressure, ECG, Weight, then select memory
"1. Device user memory" then click Save.

5. You can now start data transfer.

6. For adding a person on Memory 2, go to My profile then click on **"New"**.

7. After adding a new profile, repeat Step 2 and select your device as memory **"2. Device user memory"** then click Save.