Veroval®

DS 22

EN - Infrared fever thermometer for ear and forehead temperature measurement
Instructions for use2-21

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Warranty Certificate / Garanti belgesi / شهادة الضمان / 保固書..... 79

Dear customer,

We are pleased you decided to purchase a HARTMANN fever thermometer. The Veroval® DS 22 is a quality product for measuring human body temperature in the ear and on the forehead. The thermometer is ideally suited for measuring the temperature of children over 6 months of age and can also be used on adults. When used correctly, the device provides fast and precise measurement of body temperature in a manner that is very comfortable.

We wish you all the best for your health.

Please read these instructions carefully before first use. Measuring body temperature correctly depends on the appropriate use of the device. Otherwise, the measured values will be incorrect.

These instructions describe the individual steps in temperature measurement using Veroval® DS 22 and contain important and helpful recommendations to determine body temperature reliably. Please store these instructions for use carefully and ensure other users also have access to them.

Veroval® DS 22 is a fever thermometer that uses infrared technology to measure body temperature. Veroval® DS 22 offers faster temperature measurement compared to conventional

fever thermometers. Moreover, oral, rectal or axillary (under the armpit) measurements of body temperature are no longer needed.

Use of the supplied single-use protective caps ensures that taking a temperature in the ear is especially hygienic and the risk of transferring bacteria or viruses is greatly minimised.

The high measuring accuracy of Veroval® DS 22 has been proven in clinical studies.

Included in delivery:

- Fever thermometer with forehead measuring attachment
- 2 × 1.5 V AAA batteries
- 10 single-use protective caps
- Storage box
- Instructions for use with warranty certificate

Supply articles:


Veroval® PC 22 single-use protective cap set (contents: 20 units)

Art. no. 925481

Table of contents

	Page
1. Device and display description	4
2. Signs and symbols.....	5
3. Advantages of the thermometer	6
4. General information on body temperature.....	6
5. Safety instructions	7
6. Initial operation of the device.....	9
7. Settings	9
7.1 Setting time and date	9
7.2 Activating/disabling key sounds.....	11
8. Measuring body temperature	11
8.1 Measuring ear temperature	11
8.2 Measuring forehead temperature.....	14
8.3 Displaying saved measured values.....	16
9. Cleaning and care of the device	16
10. Batteries and disposal notes.....	17
11. Error messages	17
12. Warranty conditions	18
13. Contact details for customer queries	19
14. Technical data	20

1. Device and display description

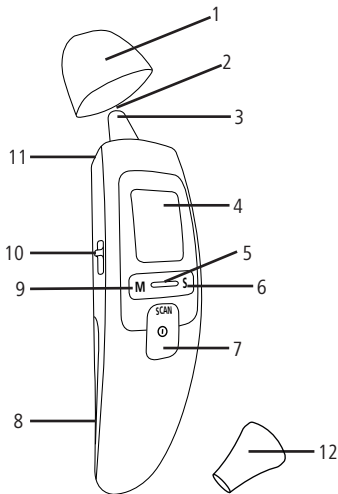- 1 Forehead attachment
- 2 Measuring sensor
- 3 Thermometer tip for taking ear temperature
- 4 Display
- 5 Fever indicator
- 6 S key to switch the sound function on and off and to adjust thermometer settings
- 7 SCAN button, ON/OFF button to switch the thermometer on and off and to start measuring
- 8 Cover for the battery compartment
- 9 M key to display the saved measured values and confirm the device settings
- 10 Mode switch for selecting measurement method
- 11 Eject switch
- 12 Single-use protective cap for taking ear temperature

2. Signs and symbols

The following icons are used in these Instructions for Use and on the thermometer:

Consult instructions for use

Consult instructions for use

Please note

IP22

Protected against penetration by solid objects with a diameter > 12.5 mm.

Protected against water drops falling at an angle of up to 15° to the monitor.

Temperature limit

Humidity limitation

Protection against electric shock

Dispose of packaging in an environmentally responsible way

Dispose of packaging in an environmentally responsible way

Cardboard recycling code

Icon for the marking of electrical and electronic equipment

Labelling in accordance with Medical Devices Directive 93/42/EEC

Labelling in accordance with Medical Devices Directive 93/42/EEC

Do not re-use / intended for single use only

Direct current

Manufacturer

EC|REP Authorised representative in the European Community (EC)

LOT Batch code

REF Catalogue number

SN Serial number

3. Advantages of the thermometer

Two measurement methods - ear and forehead temperature measurement

Verova® DS 22 measures temperature in the ear and on the forehead. Select between the two modes according to your preference. You can set it to take either ear or forehead temperature readings by using the mode switch on the side of the thermometer.

Rapid measurement using infrared technology

The measuring sensor records the infrared radiation emitted from the eardrum or the surface of the forehead and after one second displays the measurement result for ear temperature measurement and at the end of the scanning process it displays the highest measurement result that was detected for forehead temperature measurement.

Robust and reliable through high quality of workmanship

This device has been developed in accordance with the HARTMANN quality assurance guidelines. Designed to meet customer needs, it meets high stability standards.

4. General information on body temperature

The human body regulates the body temperature to a target value and it can fluctuate by up to 1°C over the course of a day. Throughout the course of life, your average body temperature can decrease by up to 0.5°C. Moreover, the temperature inside the body (core temperature) and the surface temperature on the skin have different values. Body temperature measured in the ear or on the forehead can produce a reading that is different from an oral, rectal or axillary temperature measurement. It therefore makes no sense to compare those values. Measuring a temperature in the ear and on the forehead can provide results that differ because the forehead temperature depends more on external influences than the temperature of the eardrum.

In addition, forehead temperature is also more dependent on external influences than rectal, oral or axillary temperature measurement.

In the case of several successive measurements, slightly varying values are obtained within the range of the measuring error tolerance, as a rule.

Ensure you measure body temperature regularly to determine normal ear or forehead temperature levels and then use those temperatures as a basis against which to compare measurements obtained when you suspect a fever. Note that body temperature is also influenced by ambient temperature, age, stress, sleep duration, hormones, and physical activity.

The ear or forehead temperature measurement determines core body temperature via the infrared radiation emitted by the body. Even when temperature measurement is carried out correctly, the reading can vary slightly from a rectal, oral or axillary temperature measurement using a digital thermometer.

Description	Measurement site		
	Ear / Forehead	Rectum	Mouth / Armpit
Subnormal temperature	< 35.7	< 36.2	< 35.8
Normal temperature	35.8 – 36.9	36.3 – 37.4	35.9 – 37.0

Increased temperature	37.0 – 37.4	37.5 – 38.0	37.1 – 37.4
Slight fever	37.5 – 38.0	38.1 – 38.5	37.5 – 38.0
Moderate fever	38.1 – 38.5	38.6 – 39.0	38.1 – 38.5
High fever	38.6 – 39.4	39.1 – 39.9	38.6 – 39.5
Very high fever	39.5 – 42.0	40.0 – 42.5	39.6 – 42.0

5. Safety instructions

- This device is designed to measure human body temperature in the ear and on the forehead only. Do not use this thermometer to take temperature measurements on any other parts of the body.
- The manufacturer is not liable for injury or damage resulting from improper handling.
- Portable and mobile high-frequency and communication equipment can affect this device. More detailed information is provided at the end of these instructions for use.
- Using the thermometer on different people may not be suitable if certain acute infectious diseases are present, because germs may be transferred from one person to another,

despite mandatory disinfecting by cleaning and wiping. Ask your treating doctor if you are unsure.

- Ensure the device is not left unattended near small children or people who cannot operate the device unaided. It presents a choking hazard. Children could swallow small parts.
- Replace the single-use protective cap after each use when taking ear temperature measurements to ensure a precise measurement and to prevent cross-contamination. A single-use protective cap is not needed for forehead temperature measurement.
- The ear thermometer may only be used with single-use protective caps by HARTMANN as other single-use protective caps may cause measuring errors. Once the single-use protective caps originally supplied with the device have been used up, please contact your specialist dealer or customer service to purchase replacement products.
- The thermometer is NOT waterproof. Please avoid direct contact with water or any other liquids.
- Do not expose the device to hard knocks or vibrations. Do not use the device if it is obviously damaged.
- Please ensure prior to every measurement that the lens of the sensor is clean, free of grease and undamaged.
- Please comply with the storage, transport and operating conditions defined in Chapter 14 - Technical data. Protect the device from dirt and dust, extreme temperatures, sunlight and liquids.
- For cleaning, please follow the instructions in Chapter 9 - Cleaning and care of the device.
- The device must not be opened, dismantled or repaired by the user.
- Do not interpret measured values yourself and do not use them to self-prescribe treatment.
- In addition to the actual temperature value, also inform your doctor whether it was taken from the forehead or the ear.
- Self-diagnosis based on these temperature readings or administering treatment on their basis may carry an unacceptable medical risk or even worsen the symptoms. Follow the instructions provided by your doctor.
- Using the device does not replace medical treatment in any way.
- A high temperature or persistent fever requires medical treatment, especially in small children. Please consult your doctor.

Notes on electromagnetic compatibility

- The device is suitable for use in all environments listed in these Instructions for Use, including domestic environments.
- It may only be possible to use the device to a limited extent in the presence of electromagnetic disturbances. This could result in issues such as error messages or the failure of the display/device.
- Avoid using this device directly beside other devices or stacked on top of other devices, as this could result in faulty operation. However, if it is necessary to use the device in the manner described above, this device and the other devices must be monitored to ensure that they are working properly.
- The use of accessories other than those specified or provided by the manufacturer of this device may result in increased electromagnetic emissions or a decrease in the device's electromagnetic immunity and lead to faulty operation.
- Failure to comply with the above can impair the performance characteristics of the device.

Measurement function check

We recommend carrying out a metrological check once a year for devices in professional use, for example, in pharmacies,

medical practices or hospitals. National regulations as specified by the legislator by federal law are to also be observed. Measurement function checks can be carried out by either the competent authorities or authorised maintenance providers at the user's expense.

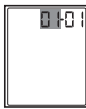
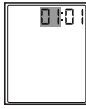
6. Initial operation of the device

Batteries are included in delivery and already inserted in the device. A contact-prevention strip sits between the batteries and the device to ensure the batteries are still at full capacity when the device is used for the first time. Please remove the contact-prevention strip carefully from the closed battery compartment. Veroval® DS 22 is now ready for operation. The Settings menu comes up automatically.

7. Settings

7.1 Setting time and date

You need to set the time and date before you start taking measurements. This will allow you to assign the saved measurements later.

	<p>1. Press the SCAN key to switch on the thermometer.</p>		<p>5. The day appears on the display. Select the current day using the S key. Confirm using the M key.</p>
	<p>2. Hold down the S key for three seconds. The year setting appears on the display.</p>		<p>6. The hour appears on the display. Select the current hour using the S key. Confirm using the M key.</p>
	<p>3. Select the correct year using the S key. Confirm using the M key.</p>		<p>7. The minutes appear on the display. Select current minute using the S key. Confirm using the M key.</p>
	<p>4. The month appears on the display. Select the current month using the S key. Confirm using the M key.</p>		<p>8. The display shows „OFF“. The thermometer switches off automatically after 2 seconds.</p>

7.2 Activating/disabling key sounds

You can activate or disable the sounds of the keys on the thermometer. If key sound is activated, a beep sounds every time you press a key and after each measurement.

To activate key sound, press the S key briefly with the thermometer switched on. A loudspeaker icon () appears on the display. To disable key sound, press the S key again. The loudspeaker icon disappears.

8. Measuring body temperature

Veroval® DS 22 can measure the temperature of both the forehead and the ear. Both types of measurement can be used to measure your own body temperature (self-measurement) and to measure the body temperature of another person (other person measurement).

Important information to prevent incorrect measured values:

- Please note that the thermometer and the person whose temperature is to be measured should be in a moderately warm room for at least 30 minutes.
- Do not measure immediately after showering, swimming, etc. or when the ear or forehead are wet.

- Avoid eating, drinking or exercising before taking a temperature measurement.
- In the following situations we recommend you carry out three temperature measurements and take the highest measurement as the result:
 - For children below the age of three with a weakened immune system (particularly if presence or absence of fever is significant).
 - If the user is not familiar with the device and until he or she obtains consistent results.
 - If measured values are suspiciously low.
 - If the measured temperature is doubtful and inconsistent with the patient's condition it is advisable to repeat the measurement after several minutes. Please also refer to Chapter 9: Cleaning and care of the device. It is also advisable to use another, independent method of temperature measurement and/or to consult a doctor.

8.1 Measuring ear temperature

The reading can be carried out in the right or the left ear. Ensure the mode switch is set to Ear temperature measurement. The thermometer measures the infrared radiation emitted from the eardrum and the auditory canal. This infrared radiation is captured by the sensor and converted to temperature values. Best values are obtained if infrared radiation from

the middle ear (the eardrum) is captured. Radiation that is emitted by the surrounding tissue gives off lower values.

If the forehead attachment is in place for Forehead temperature measurement, please remove it by simply pulling it off.

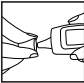
- In some people, temperature readings in the left ear will differ from readings taken in the right ear. To identify changes in temperature, always take the reading in the same ear on the same person.
- Children may only measure in-ear temperature under the direct supervision of an adult. Taking a child's ear temperature is generally possible from the age of 6 months onwards. In children under 6 months of age, the auditory canal is still very narrow so that the temperature of the eardrum therefore often cannot be determined precisely and measurement results displayed are usually too low.
- The measurement must not be carried out in an inflamed ear (e.g. pus or discharge from the ear), after possible ear injuries (e.g. eardrum injury) or during the healing phase after a surgical procedure. Please contact your treating physician if this is the case.

Please note when taking a temperature in the ear:

- Make sure you are using the single-use protective caps with the device when taking the ear temperature.
- If a single-use protective cap is not used, earwax may accumulate on the measuring sensor, which may lead to inaccurate temperature measurement or even transfer of an infection between different users. It is imperative therefore that the measuring sensor is always clean when taking a temperature. Directions for cleaning are provided in the instructions in Chapter 9 - Cleaning and care of the device.
- Please ensure the sensor is pointing directly at the eardrum and not at the auditory canal.
- After cleaning the measuring sensor, please wait 15 minutes before taking the next measurement so that the thermometer can reach the required operating temperature.
- If you have rested your head on one ear, the temperature in that ear may be briefly elevated.

	<p>1. Press the SCAN key to switch on the thermometer.</p> <p>Set the mode switch on the side to . Ensure the Ear icon  is displayed.</p>
	<p>2. Place a new, clean protective cap on the tip of the thermometer's sensor until the protective cap icon is shown on the display.</p>
	<p>3. Ensure the sensor tip and the outer ear canal (auditory canal) are clean. Since the auditory canal is slightly curved, you need to gently pull the ear backwards before inserting the sensor tip so that the tip can be pointed directly at the eardrum.</p>

	<p>Positioning children less than one year old</p> <ul style="list-style-type: none"> Place the child on their side so that one ear faces upwards. Gently draw the ear backwards before carefully inserting the tip of the sensor into the ear.
	<p>Positioning a child older than one year</p> <ul style="list-style-type: none"> Stand behind the child, slightly to one side. Gently pull the ear upwards and backwards before carefully inserting the tip of the sensor into the ear.
	<p>4. Carefully insert the sensor tip and press the SCAN key for one second. The "°C" icon flashes on the display while taking the temperature.</p>

	<p>5. After about 1 second, the measurement result appears and the backlighting illuminates for 5 seconds.</p> <ul style="list-style-type: none">- If temperature taken is less than 37,6°C, the fever indicator lights up in green.- If the temperature taken is 37,6°C or higher, the fever indicator lights up in red. <p>If you do not press any key for one minute, the thermometer switches off automatically.</p>
	<p>6. Clean the thermometer tip after every use and dispose of the single-use protective cap. To do so, use a soft cloth or a cotton bud that can be moistened with disinfectant, alcohol or warm water.</p>

8.2 Measuring forehead temperature

The thermometer measures the infrared radiation emitted by the skin on the forehead, by the temple area and by the surrounding tissue. Ensure the mode switch is set to Forehead temperature measurement. Infrared radiation is captured by the sensors and converted to temperature values. The most accurate values are obtained when you scan the entire area from one temple to the other.

Always use the forehead attachment for forehead temperature measurements. Do not use a single-use protective cap intended for ear temperature measurements when taking the forehead temperature as this will distort the result.

Please note when taking a temperature on the forehead:

- Note that forehead and temples must be free of sweat and cosmetics and that consumption of vasoconstrictor medications and skin irritations can distort the result when taking forehead temperature.
- Please note that forehead temperature can be affected by being outdoors for longer periods (e.g. by the cold during winter or by intensive sunshine during the summer) as well as by wearing headgear. It is recommended to measure forehead temperature only after having rested at room temperature for at least 30 minutes.
- We recommend you always measure over the same region, as the values displayed could otherwise vary.

- It is essential to extend the scanning movement to include the temple and temple area to ensure the temporal artery is included in the measurement.

	<p>1. Press the SCAN key to switch on the thermometer.</p> <p>Switch the mode button on the side to . Ensure the Forehead icon is displayed.</p>
	<p>2. Place the sensor head of the device with forehead attachment on the temple. Hold the SCAN key down and then slowly and evenly (taking about 5 seconds) move the thermometer over the forehead to the other temple. The „°C“ icon flashes on the display while taking the temperature.</p>

	<p>3. After you have moved the thermometer over the forehead, release the SCAN key. The temperature appears and the background illumination lights up for 5 seconds.</p> <ul style="list-style-type: none"> - If the temperature taken is less than 37,6°C, the fever indicator lights up in green. - If the temperature taken is 37,6°C or higher, the fever indicator lights up in red. <p>If you do not press any key for one minute, the thermometer switches off automatically.</p>
	<p>4. Clean the thermometer tip and the forehead attachment after each use. To do so, use a soft cloth or a cotton bud that can be moistened with disinfectant, alcohol or warm water.</p>

8.3 Displaying saved measured values

The device automatically saves the values of the last 10 measurements. Once the 10 memory positions are occupied, the oldest temperature reading is overwritten. Saved temperatures can be accessed as follows:

	1. Press the SCAN key to switch on the thermometer.
	2. Press the M key to display saved temperatures. The latest measured value, its memory position number and the Forehead or Ear icon appear on the display (37.0), alternating with the corresponding date and time.
	3 To switch between saved individual temperatures, press the M key. If you do not press any key for one minute, the thermometer switches off automatically.

9. Cleaning and care of the device

The measuring sensor is the most important and most sensitive part of the device. To ensure the most accurate measurement, the sensor must always be clean and undamaged.

Use only a soft, moistened cloth or 70% alcohol (isopropyl alcohol) to clean the measurement tip and the sensor. Please do not use any aggressive cleaning agents or solvents, as they can damage the device and cloud the display. Make sure no liquid penetrates the inside of the thermometer. After cleaning the measuring sensor, wait until the cleaning liquid has dried off completely before taking a measurement. Keep the thermometer in the storage box to protect it from environmental influences.

10. Batteries and disposal notes

- The two premium quality batteries included in the delivery guarantee at least 1,000 measurements. Use only premium quality batteries (see specifications in Chapter 14 - Technical data). If you use less efficient batteries, we can no longer guarantee 1,000 measurements.
- Never mix old and new batteries or batteries from different manufacturers.
- Remove exhausted batteries immediately.
- Remove the batteries to prevent leakage if you do not intend to use the monitor for a longer period of time.
- Please respect the environment and do not dispose of batteries in household waste! Hand them in at collection centres or at municipal special waste recycling centres.

This symbol on products and/or accompanying documents means that used electronic products must not be disposed of with regular household waste.

Battery replacement

This device is equipped with two alkaline batteries type LR03 (AAA). Replace flat batteries with two new LR03 batteries as soon as the this battery icon appears on the display .

Remove the screw of the battery cover on the rear of the device using a Phillips head screwdriver. Now insert two AAA batteries so that the positive (+) and negative (-) poles on the batteries match the '+' and '-' signs of the markings in the battery compartment. If polarity is not correctly aligned, the device will not function and the batteries may leak. After changing the batteries, carefully close the battery cover again. The last measured value remains in the memory when you change batteries.

11. Error messages

Veroyal® DS 22 is a clinically-tested premium quality product. Nevertheless, error messages may be displayed if, for example, the ambient temperature is too high or too low, if the measured temperature is outside the range of human body temperature, if the batteries are flat or, in very rare instances, as a result of a system error.

Error message	Cause	Remedy
CAP	When carrying out a measurement in Forehead mode, the single-use protective cap for ear temperature measurement is detected.	Ensure the device is used in the appropriate measuring mode and that a single-use protective cap is not used to take forehead temperature.
H _i	The temperature measured is over 43°C	Use the thermometer only within the temperature ranges indicated. Clean the sensor tip if necessary. In case of repeated error messages, contact your specialist dealer or customer service.
Lo	The temperature measured is less than 34°C	
Er 1	The operating temperature is not between 10°C and 40°C.	Use the ear and forehead fever thermometer only within the temperature ranges indicated.

Er 2	The thermometer is positioned incorrectly or is not stable.	Note correct positioning given in the instructions for use and repeat the measurement.
Er 3	The thermometer does not function correctly.	Insert new batteries. If the error message persists, please contact customer service.
	The batteries are flat.	Insert new batteries into the thermometer.

12. Warranty conditions

- We provide a 3-year warranty from date of purchase on this premium quality ear and forehead fever thermometer, subject to the conditions listed below.
- Warranty claims must be made during the warranty period. The purchase date must be documented by an appropriately completed and stamped warranty certificate or proof of purchase.
- HARTMANN will replace or repair free of charge any faulty device components caused by material faults or manufacturing errors within the warranty period. This does not extend the warranty period.

- The device is intended for the purpose described in these Instructions for Use only.
- The warranty does not cover damage resulting from improper use or unauthorised interference. Accessory parts that are subject to wear and tear (batteries, single-use protective caps, forehead attachment, storage box, packaging, etc.) are excluded from the warranty.

Claims for compensation are limited to the value of the merchandise; compensation for subsequent damage or injury is expressly excluded here.

For all warranty claims, please send the device, forehead attachment and storage box, along with the completed and stamped warranty certificate or proof of purchase, directly to your dealer or via your dealer to the Customer Service Department for your country.

13. Contact details for customer queries

ZA

HARTMANN South Africa
Epsom Avenue, Northriding,
2169 Johannesburg
www.veroval.info
Tel. +27 860 4278 6266
phzahelpdesk@hartmann.info

14. Technical data

Model	Veroval® DS 22
Type	DS 22
Measuring range	Ear / Forehead mode: 34°C – 43°C
Laboratory measuring accuracy	Ear mode: ±0.2°C for 35.5°C – 42°C, outside this measurement range ±0.3°C Forehead mode: ±0.3°C for 35°C – 42°C
Time interval between two measurements	At least 5 seconds
Clinical measuring accuracy	Clinical deviation: 0.06°C Clinical repeatability: 0.13°C Limits of agreement: 0.94°C
Measurement unit	°Celsius (°C)
Operating conditions	10°C – 40°C at a relative humidity of 15% - 85% (non-condensing)
Storage and transport conditions	-25°C – 55°C at a relative humidity of 15% - 95% (non-condensing)

Dimensions	155 x 42 x 35 mm
Weight	60 g (without batteries)
Battery	2 × 1.5V AAA (LR03)
Memory	10 measurements
Optical fever alarm	Red status display at a temperature ≥ 37.6°C and green status display at a temperature < 37.6°C
Measurement time	About 1 second or the duration of the scanning procedure for forehead temperature measurement
Smallest display unit	0.1°C
Protection against harmful ingress of water or solid materials	IP 22
Expected service life	10.000 measurements
Battery capacity:	Approx. 1.000 measurements
Automatic switch-off	After approx. 1 minute
Serial number	On the type plate

Permissible protective caps	Veroyal® PC 22 single-use protective caps
-----------------------------	---

Legal requirements and guidelines

The device complies with the EU Medical Devices Directive 93/42/EEC, the German Medical Devices Act, ASTM E 1965 - 98, the European standard EN 12470-5: Clinical thermometers – Part 5: Performance of infrared ear thermometers (with maximum device) as well as the European standard EN 60601-1-2 (in accordance with CISPR11, IEC 61000-4-2, IEC 61000-4-3), and is subject to special precautionary measures regarding electromagnetic compatibility. Please note that portable and mobile high-frequency and communication devices can affect this device. More detailed information is provided at the end of these instructions for use or can be requested by contacting the customer service address provided above.

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Made under the control of PAUL HARTMANN AG
JOYTECH Healthcare Co., Ltd.
Medical Device Safety Service GmbH